

PHONE ADAPTER

Phone calls – Daily use

USER GUIDE

Intro

This user guide explains how the SoundGate is used for phone conversations.


The Phone adapter provides a wireless connection between your SoundGate and your home phone. It supports both incoming and outgoing phone calls. The SoundGate enables hands free operation.

First of all make sure that the Phone adapter is correctly connected to both the phone land line and power. Please refer to the installation guide.

The wireless phone application is possible within a range of approx. 10 meters (30 feet) from the Phone adapter.



SoundGate overview

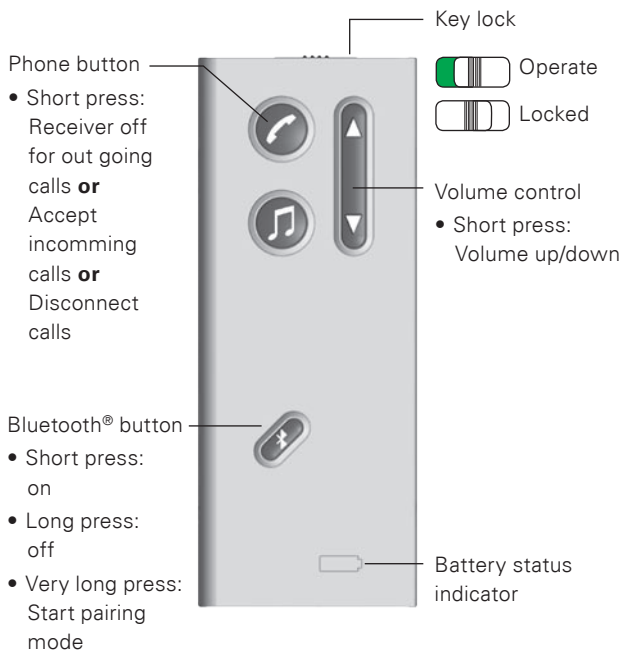

Phone calls can also be received when the keys are locked!

Incoming calls

Before talking

Wear the SoundGate around your neck using the neck strap. Make sure that the Bluetooth® function on the SoundGate is switched on. You must be within 10 meters (30 feet) of the Phone adapter and its red status indicator must be steadily illuminated.

Incoming calls

A ringing tone will be heard in the hearing instruments and the green rim light around the phone button will flash.



Answer

Give a short press to the SoundGate phone button. The SoundGate will answer the call and the green rim light will stop flashing and will be steadily illuminated. The call is now directed into the hearing instruments. Speak normally, as if the other person was sitting in front of you. The red status indicator on the Phone adapter will flash during the call.




Status

Hang up

Give a short press to the SoundGate Phone button. The SoundGate will hang up the call and the green rim light will turn off.



The red status indicator on the Phone adapter will now be steadily illuminated.


Status

Outgoing calls

Make an outgoing call

Give a short press to the SoundGate phone button or shortly press the red phone button on the Phone adapter.

You will hear a dial tone in the hearing instruments. The green rim light will be steadily illuminated.

Pick up the receiver or press the call button on the phone and quickly dial the phone number. Once the call is finished replace the receiver or press end call on the phone.

The red status indicator on the Phone adapter will flash during the call.



Status

Hang up

Give a short press to the SoundGate phone button. The SoundGate will disconnect the call and the green rim light will turn off.

The red status indicator on the Phone adapter will now be constantly illuminated.



Status

Make sure that the phone is turned off again, or the receiver is replaced!

Notice: Your voice level to your conversation partner is automatically adjusted by the SoundGate, but you should always speak loudly and clearly. If necessary speak directly into the microphone of the SoundGate.