

SOUNDGATE

Connected to the
World with SoundGate



bernafon 
Your hearing • Our passion

Options for being connected are greater than ever. Through a variety of electronic devices, we connect to our family, friends and colleagues, anytime or anywhere. We can also listen to music, watch videos, hear the latest news, and more. Your hearing aids, together with Bernafon's SoundGate, expand the possibilities for being connected.



Where other wireless devices limit connection to one audio source or another, SoundGate gives you the freedom to listen with your hearing aids to a number of wired and wireless audio sources. It is discreetly designed, comfortable to handle, and easy to use. It has a built-in microphone, an integrated Bluetooth® receiver, and user-friendly controls. SoundGate keeps you connected with ease and style.



Bernafon's SoundGate enables you to connect and communicate anytime, and from anywhere.

Everyday listening situations may be challenging with even the best hearing aids. But with simple to use solutions, Bernafon gives you the freedom to listen comfortably.



Mobile Phone Streaming


Connect and talk wirelessly using your SoundGate as a Bluetooth® interface between your mobile phone and your hearing aids.

Easy Communication with the SoundGate Mic

Use this discreet microphone for one-to-one communication in very noisy environments and understand speech more easily with less effort.

Clear and Sharp TV Listening

Let the sound come directly from your TV to your hearing aids using SoundGate with Bernafon's TV Adapter 2.

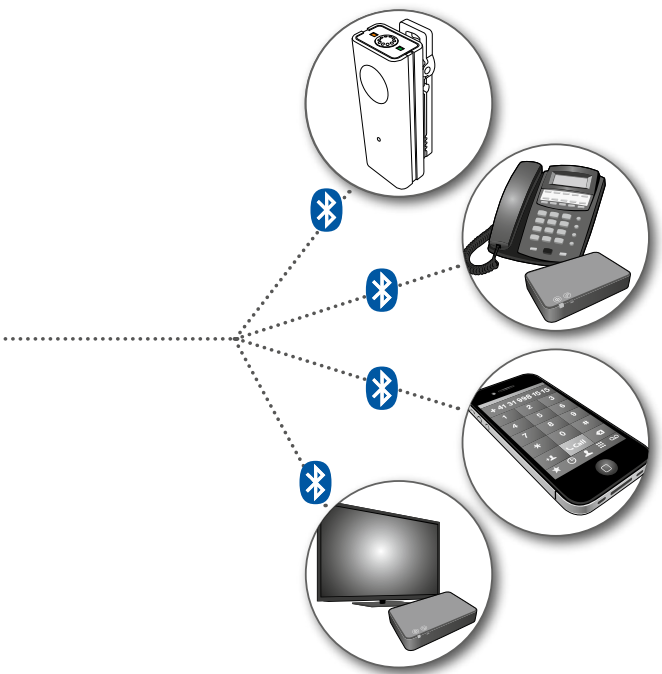

Headset for Music or GPS

SoundGate makes it simple for you to listen to your music files or GPS instructions wirelessly through your hearing aids via Bluetooth®.

Hands-Free Landline Calls

Enjoy wireless landline calls and greater talk-time at home or at work with your SoundGate and Phone Adapter 2.

Telecoil

With SoundGate, connect easily and automatically to looped telecoil systems in theaters, classrooms or places of worship.

Wireless Mobile Phone Connection with SoundGate

SoundGate works as an interface between your hearing aids and Bluetooth®-enabled mobile phones. You never know when that important call will come through. But your SoundGate will be waiting, primed and ready. And, it allows you to talk wirelessly and hands-free without using the mobile phone itself.

With SoundGate, you can also listen to music that is stored on your mobile phone or media player through your hearing aids.

The SoundGate charging station is easy to use and ensures that the device is powered up when needed.

Looped Telecoil Systems and SoundGate

Many theaters, classrooms and places of worship are now equipped with pre-installed induction loop systems. Listen to audio in these places with SoundGate's integrated telecoil receiver. Audio signals transmit wirelessly without the need for a separate device, or even a special listening program in your hearing aids.

Enjoy greater mobility and greater flexibility on the go with SoundGate.

Enjoy Ease of Communication with the SoundGate Mic

Bernafon offers the SoundGate Mic for even greater success in challenging listening situations.

This discreet microphone can be worn by the person you are speaking with, or who is making a presentation, or who is giving a speech. Coupled with the SoundGate (SoundGate 3 or newer versions), the speech signal is transferred directly to your hearing aids.

It can really help you understand what is being said, with even less effort. This is especially so when there is background noise (e.g., in a restaurant) or reverberation in the room (e.g., in an auditorium).



Home Sweet Home

Whether you're in the mood for a relaxing evening in front of the TV or having a long phone conversation with family or friends, SoundGate will help make every word crystal clear.

Wireless TV Connection with SoundGate

TV listening has never been easier. With Bernafon's small and portable TV Adapter 2, sound is transmitted clearly from your television to your hearing aids via SoundGate.

The best part about the TV Adapter 2 is that you control the volume coming from the TV to your hearing aids via SoundGate separately.

Not everyone has to listen at the same volume.





Wireless Phone Connection with SoundGate

Conveniently connect landline telephone calls to your hearing aids with Bernafon's Phone Adapter 2.

Phone Adapter 2 and SoundGate alert you to incoming calls, transmit the signal, and essentially turn your hearing aids into a pair of wireless headsets. This allows you to enjoy hands- and hassle-free calls from your landline phone too.

Being connected is easier and more enjoyable than ever with Bernafon's wireless accessories.

SoundGate App – Smart and Discreet

With the SoundGate App, Bernafon goes beyond the usual controls of a hearing aid. Use your iPhone® or Android™ smartphone together with the SoundGate to switch programs, turn the volume up or down, and connect to multiple devices. Changing programs and changing the volume has never been so easy and certainly not so discreet.¹⁾

Download the SoundGate App for free. Just visit the App Store or Google Play™ and search for “Bernafon SoundGate” and then click to install.



Apple, the Apple logo, and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Android and Google Play are trademarks of Google Inc.

¹⁾ The SoundGate App for iPhone® runs on iPhone 6 Plus, iPhone 6, iPhone 5s, iPhone 5c, iPhone 5, iPhone 4s, and iPhone 4, and requires Bernafon wireless hearing aids and SoundGate 3 with firmware 3.0 or later.

The SoundGate App for Android™ smartphones requires Bernafon wireless hearing aids and SoundGate 3 with firmware 3.1 or later.



Hearing and understanding are basic human needs. Hearing problems and the resulting social consequences represent a challenge to society. We believe in a world where people with restricted hearing can communicate again thanks to advanced technology.

Visit **www.bernafon.com** for more information about hearing and hearing loss.

United States of America

Berna fon, LLC
2501 Cottontail Lane, Suite 102
Somerset, NJ 08873
Phone +1 888-941-4203
info@bernafon-us.com

SWISS 
Engineering

berna fon 
Your hearing · Our passion

Berna fon Companies: Australia · Canada · China · Denmark · Finland · France · Germany · Italy · Japan · Korea · Netherlands · New Zealand · Poland · Spain · Sweden · Switzerland · Turkey · UK · USA