

5 tips to get the best out of your rechargeable hearing aids

- 1 Charging** Always fully charge the batteries, until the light in the charging dock is solid green – it takes approximately 7 hours.
- 2 Out of power** If the batteries are drained and the hearing aids have switched off, never try to get more out of the batteries by opening and closing the drawer. Either:
 - Place the hearing aids in the charging dock and ensure it is powered on
 - Insert disposable batteries and keep the rechargeable batteries safe for charging later
- 3 Not in use** If you are not using your hearing aids for an extended period of time, take the batteries out.

Note: Never leave the rechargeable batteries in the hearing aids with the battery drawers open. This will compromise battery life.
- 4 Handling** Do not keep the rechargeable batteries together with metal objects such as keys and coins.
- 5 Maintenance** Wipe off any moisture from the hearing aids or charging dock before charging.