VIDEO VRA PRO 2 SYSTEM

FEATURES & BENEFITS

Features:

- •Bluetooth Low Energy Wireless Remote
- •The most flexible system on the market, with the ability to combine or expand <u>up</u> <u>to 5 units</u> of any combination of <u>Video</u> and Traditional VRA
- •Easy Pairing with Remote
- Easy Installation
- •No Interconnecting Cables (between side reinforcers)
- •Install Kit Included in system
- •No Need for Additional Sensors or wires
- •No Frequency Overlap on Adjacent Systems
- •Our Video VRA works with our exclusive Conditioned Play Audiometry (CPA) **BIG RED BUTTON System**

ADVANTAGES OF VRA PRO

Our VIDEO VRA Pro 2 System was designed with the audiologist and the installer in mind, with install simplicity, the only cable required is power from an outlet.

Our Bluetooth Low Energy remote, once properly paired, allows for a secure connection, once paired with the tablet, it will not get frequency overlap or interference (like in other RF systems) and does not require sensors placed around the audiometric booth (like in IR system).

Our VRA Pro 2 System is the most flexible system on the market. The audiologist can combine up to 5 units of Video and/or Traditional VRA PRO 2 systems to allow up to 5 reinforcer units to be controlled from the same wireless remote.


Our System does not require any interconnecting cables between the two side reinforcers.

Our System has the ability to run the primary units from a GSI ASP and ASP-2* (*Video requires ASP interface)

Customizable Content for Videos or Pictures

CONDITIONED PLAY INNOVATIONS VRA Systems



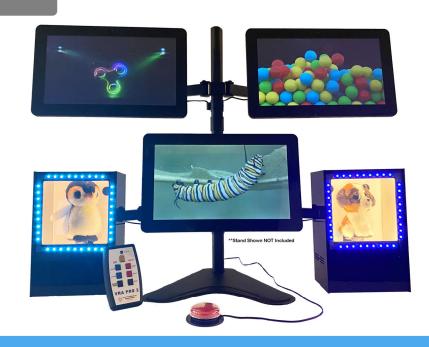

VIDEO & Traditional Combo VRA PRO 2-5 Unit System Pictured with CPIs Exclusive Big Red Button CPA System (Optional)

Consumer/Audiologist

WEBSITE: www.vrapro.com PHONE: 801-607-1800 EMAIL: sales@vrapro.com

VIDEO VRA PRO 2 SYSTEM

CONDITIONED PLAY INNOVATIONS





VIDEO VRA PRO 15.6" Two Monitor System Pictured
**Stand Shown NOT Included

Frequently Asked Questions:

Q: Can I bill for Visual Reinforcement Testing (VRA) testing?

A: CPT code 92579 is used for VRA testing

Q: Do I need a specific audiometer to use the VRA Pro?

A: No, the VRA pro is operated by an independent stand alone remote, it can be used with any audiometer. However, we do have the option of a hardwired control with the GSI ASP and ASP 2 audiometers (Optional -wired ASP or ASP 2 cable and interface required)

Q: Is the VRA Pro Infrared (IR) or Radio Frequency (RF) ?

A: It is technically RF, but uses a 2.4GHz (Bluetooth Low Energy) connection that is encrypted and secure.

Q: Will I get interference if I have multiple VRA Pro 2 systems installed in adjacent Rooms?

A:It will not have the same interference as other RF (900 MHz) systems on the market. It also does not require any IR sensors to be placed around the room, unlike all infrared systems on the market.

Q: Can I have more than just 2 side reinforcers and 1 centering display?

A: Yes, our system can operate up to 5 separate reinforcers, 2 right, 2 left, and a centering unit with the same remote control. This can be any combination of VIDEO VRA PRO 2 or VRA PRO 2 Traditional units.

Q: I already have a VRA PRO 2 Traditional System, will the Video VRA work with my same remote?

A: Yes, if you already have a Video VRA Pro 2 system, you can operate a VIDEO VRA PRO 2 and your current VRA from the same wireless remote

Q: Can I add the Conditioned Play Innovations Big Red Button CPA system to my Current VIDEO VRA?

A:The CPI Big Red Button System is exclusive to the Video VRA Pro 2 from Conditioned Play Innovations. It is available in a single unit VRA/CPA system.

Q: Do I have to run a lot of cables from each of the units, like sensor, transmitters, or IR receivers?

A:No our Video VRA only required power to be ran to the video display, the pairing with Bluetooth low energy happens directly in the application

Q: How Large is the video screen on the CPI VIDEO VRA PRO 2 System?

Q: Is the Video VRA Pro 2 able to mount on a VESA mount?

A:Yes, a swing mount is included for each monitor for our VRA PRO 2 system.

Consumer/Audiologist

WEBSITE: www.vrapro.com PHONE: 801-607-1800 EMAIL: sales@vrapro.com