

OPERATIONS MANUAL

MedRx Video Otoscope Software

Bringing Video Otoscopy To Your Computer



MedRx[®]

Let Our Image Enhance Your Image

Contents

Video Otoscope Software

Video Otoscope Software Installation	3
USB Video Capture Device Driver Installation	5
USB Camera Driver Installation	7
Software Configuration	10
Adjusting Video Quality (USB Video Capture Device)	12
Configuring the Software (USB Camera)	13
Adjusting Video Quality (USB Camera)	14
Taking Pictures.....	16
The Capture Screen	17
The Main Screen.....	18
The View Screen.....	19
Saving Images	19



www.medrx-usa.com



1200 Starkey Rd., #105, Largo FL 33771 U.S.A.

Toll Free: (888) 392-1234 • (727) 584-9600

Fax: (727) 584-9602 • Email: medrx@medrx-usa.com

The MedRx Video Otoscope Software

Software Installation

DO NOT PLUG THE USB CABLE INTO YOUR COMPUTER UNTIL INSTRUCTED TO DO SO.


The MedRx Video Otoscope software may be incorporated into any AVANT™ software package. It is also available as a stand-alone application. When bundled with the software packages, the Video Otoscope software is automatically installed. In this case, go directly to the Driver Installation section on page 5.

Stand Alone Video Otoscope Software Installation


1. Insert the MedRx Video Otoscope CD-ROM into the CD drive. Wait until the **Setup program starts**.
 - If the Setup program does not automatically start, go to **my computer**, double click on **CD**.



2. On the Setup screen, choose **Install OtoScope**.

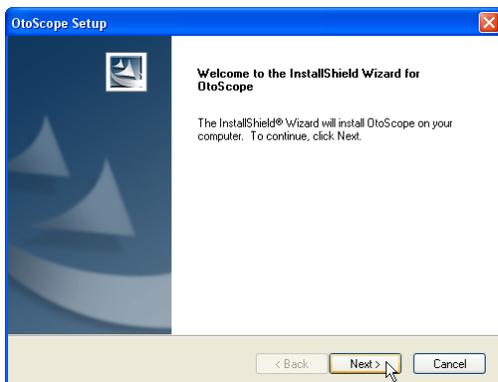


3. This is the Welcome screen.
 - To continue, Click **Next**.

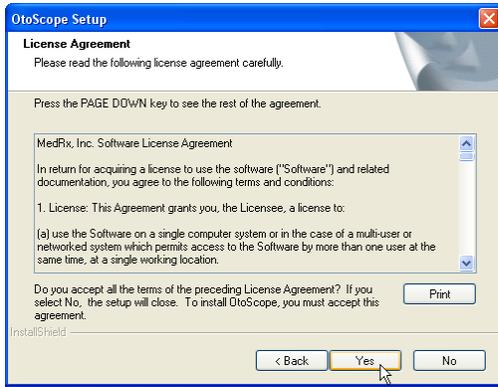


4. Read the Software License Agreement. This important document defines the acceptable usage of the Otoscope software.

After reading the Agreement, click **Yes**.

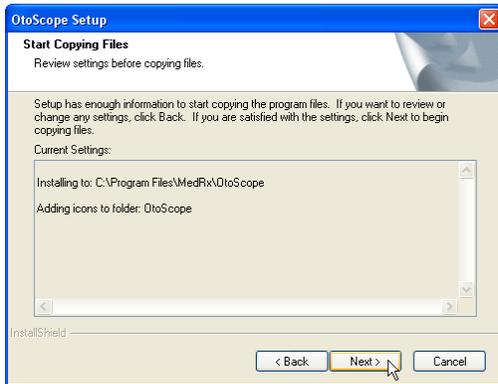

5. This screen summarizes the installation configuration.

- To continue, click **Next**.
- To make changes, click **Back**.

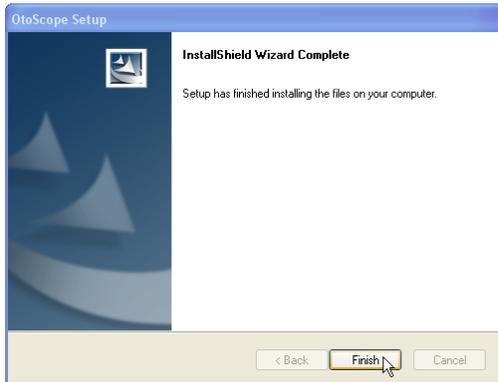


6. When the installation is complete,

- Click **Finish**.


USB Video Capture Device Driver Installation

Connect the USB cable from the USB Video Capture Device to your computer as shown below. If using the USB Camera, go to page 7.

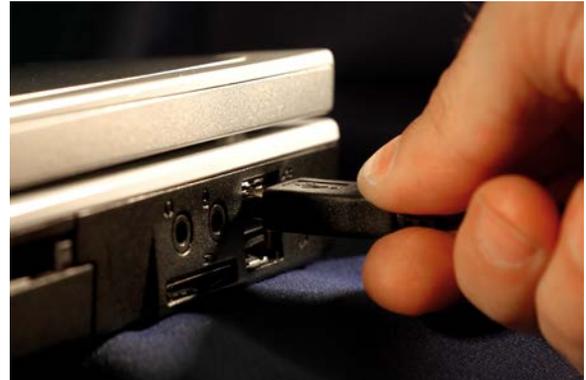


Shown Above - Typical USB Capture Cables


1. Insert the Video Driver CD-ROM into the CD drive.

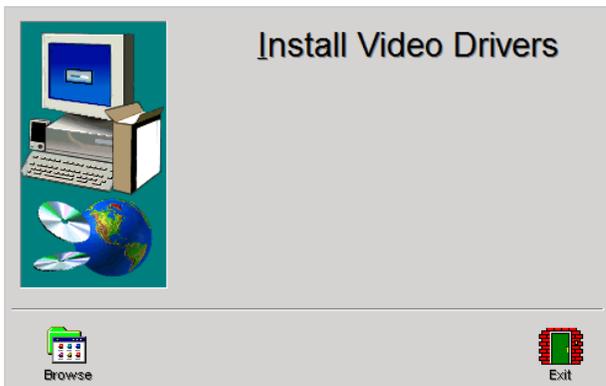


2. On the Video Driver Setup screen, click **Install Video Drivers**.

3. When this screen appears, click **Ok**.

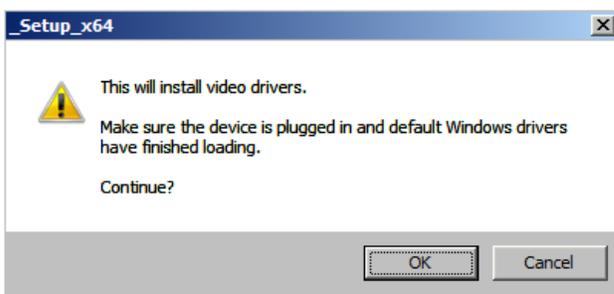


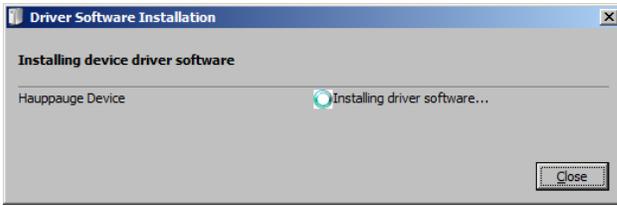

4. When this window appears and device is "Ready to use", click **Close**.

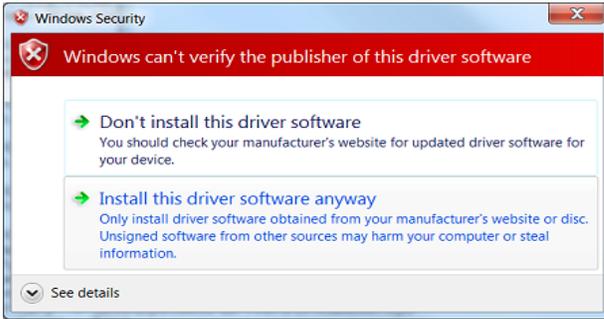

5. If this warning screen appears, click **Install this driver software anyway**.

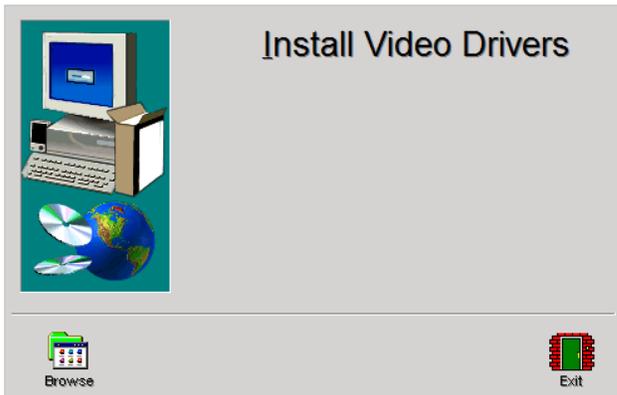


6. On Setup Window click **Exit**

USB Camera (STC-MC83) Driver Installation

Connect the USB cable from the USB Camera Device to your computer as shown below.




Shown Above – Back of USB Camera

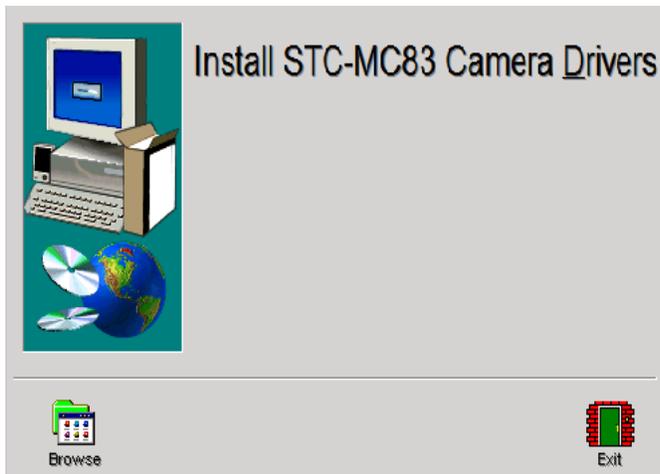


7. Insert the **STC-MC83 Driver CD-ROM** into the CD drive.
8. On the Driver Setup screen, click **Install STC-MC8 Drivers**.




9. When this screen appears, click **Ok**.

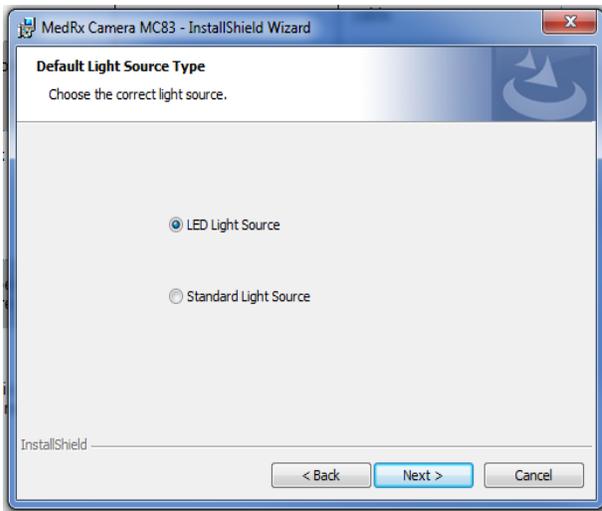


10. When this window appears choose which light source click **Next**.

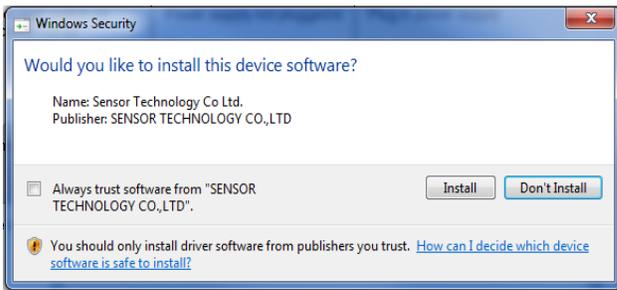

11. This screen will appear, click **Install** and check **Always Trust Software**.

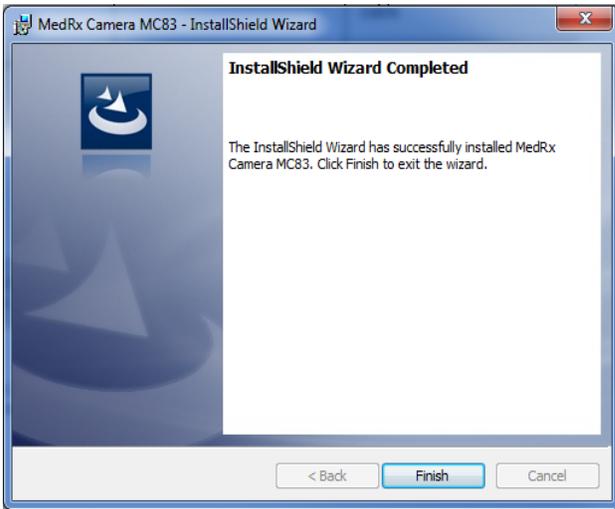

12. On InstallShield Wizard Window click **Finished.**

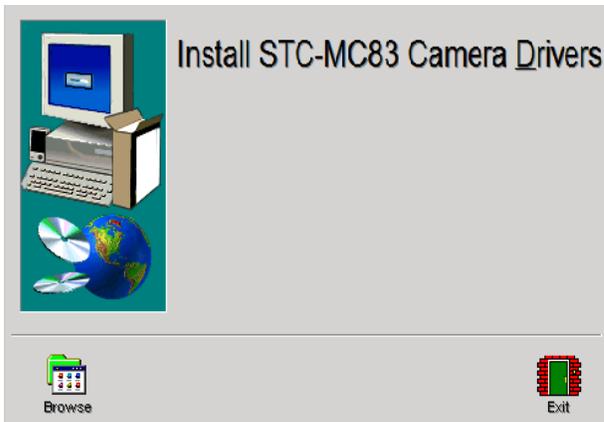

13. Installation Complete click **Exit.**

Software Configuration

The MedRx Video Otoscope software can run stand-alone, through Avant Family Device software or through the NOAH™ System or the TIMS® Office System.

Launching the Software

Stand Alone or AVANT Family

1. Double click the MedRx OtoScope icon or select Otoscope from any **AVANT Family device** shortcut on your Windows Desktop.



NOAH and TIMS

1. Launch NOAH and select **AVANT Family device** icon.




Configuring the OtoScope Software (USB Video Capture Device)

Once the drivers are installed, you must configure the software to recognize the video capture device as follows:

For Stand Alone software

From the main screen of the Otoscope software,


- Click **File**.
- Click **Video Setup**.

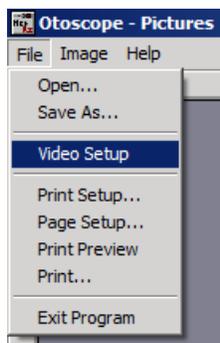


For Avant Family software

From the main screen of the Otoscope software,

- Click **File**.
- Click **Advanced Options**

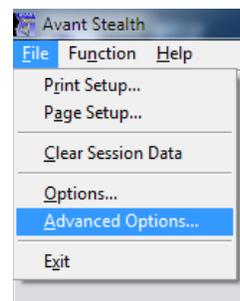


- Then Click **Video tab**

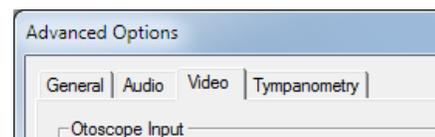


Both are continued on next page

For Stand Alone and Avant Family software

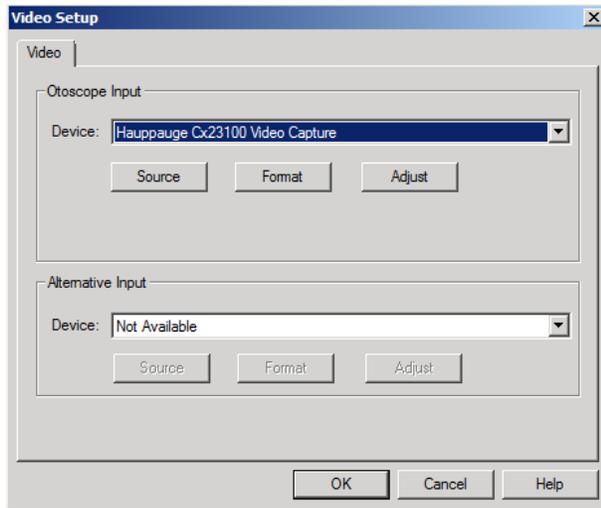


1. From the **Video Tab**:
 - Click the **Device** pull-down.
 - Select the name of your Video Capture Device
2. In the same screen, click the **Source** button.

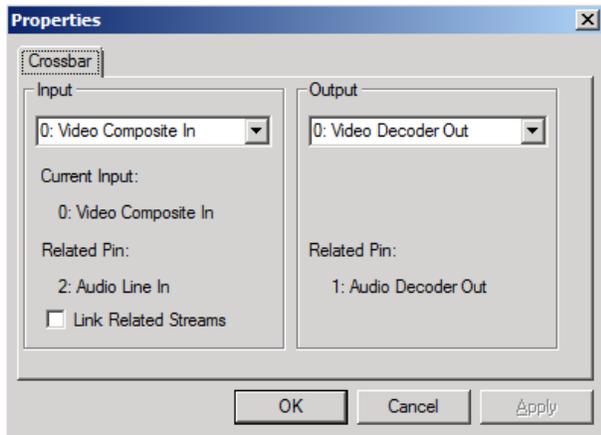


3. In the Properties window, use the pull-downs to verify the following settings:
 - **Input = Video Composite In.**
 - **Output = Video Decoder Out.**
4. Click **OK**.

Adjusting Video Quality (USB Video Capture Device)

It is sometimes desirable to adjust the video quality of the OtoScope software. To do this, follow the steps below:

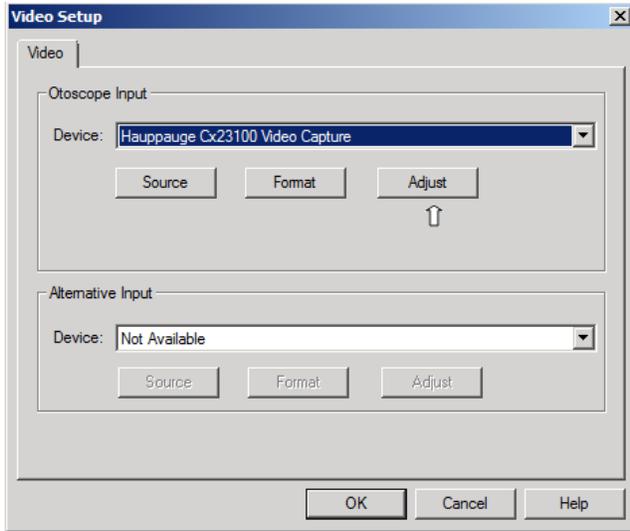

5. From the **Video Tab**:
 - Click **File**.
 - Click **Video Setup**.
 - Click the **Adjust** button.

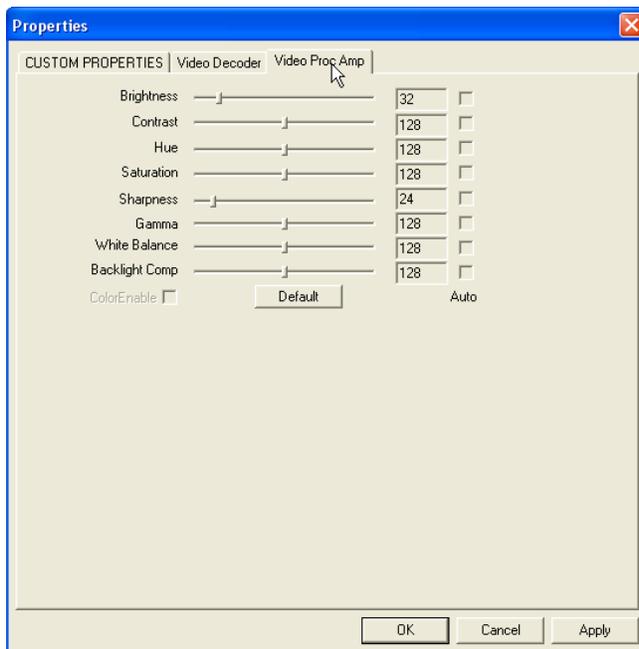

1. Click on the **Video Proc Amp** Tab.
2. Adjust the sliders as needed.
3. Click **OK** to complete.
4. Repeat as needed to optimize the image on your particular system.


Configuring the Otoscope Software (USB Camera)

Once the drivers are installed, you must configure the software to recognize the video capture device as follows:

For Stand Alone software

From the main screen of the Otoscope software,


- Click **File**.
- Click **Video Setup**.

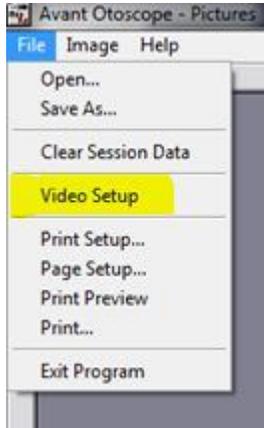


For Avant Family software

From the main screen of the Otoscope software,


- Click **File**.
- Click **Advanced Options**

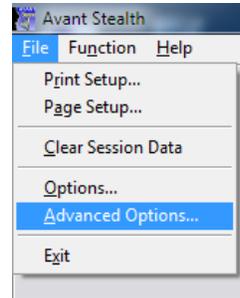

- Then Click **Video tab**

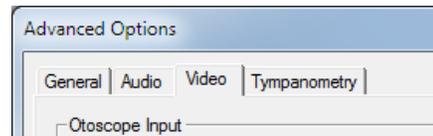


1. Select StUSBCam for the video device, click **OK**

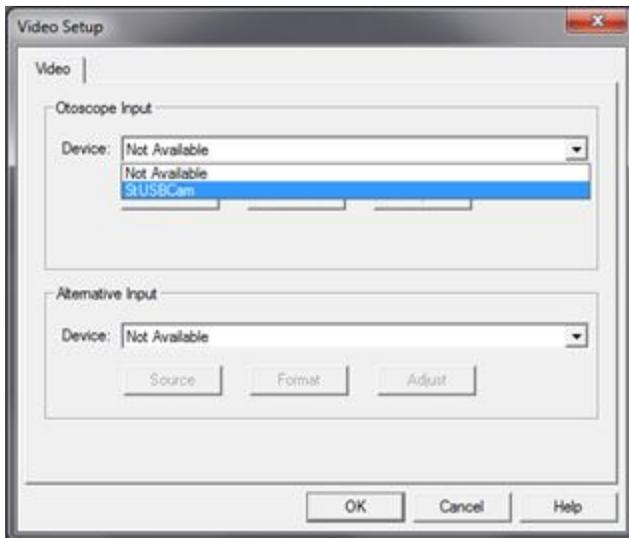


Adjusting Video Quality (USB Camera)


It is sometimes desirable to adjust the video quality of the Otoscope software. To do this, follow the steps below:

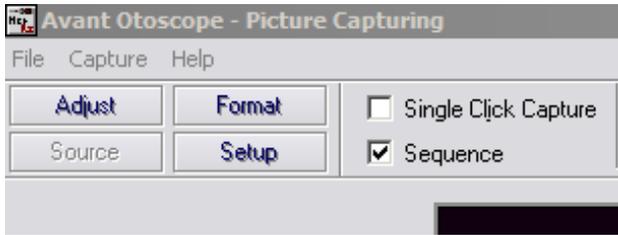

1. Click on the **Adjust** button

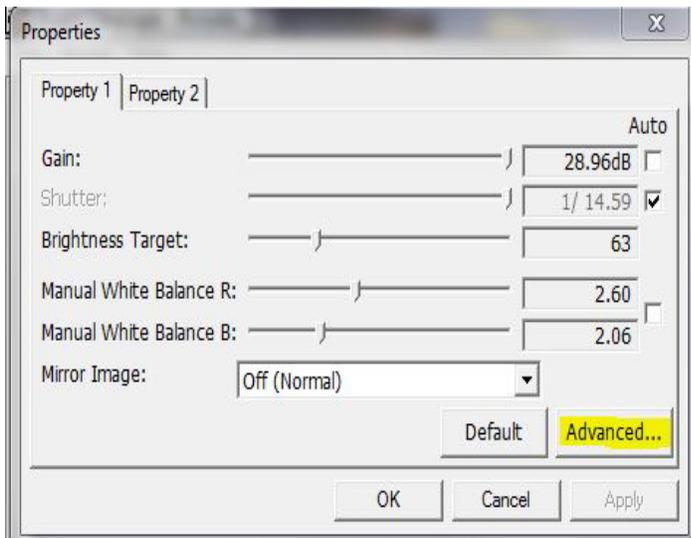


2. On the Properties screen, click on **Advanced**

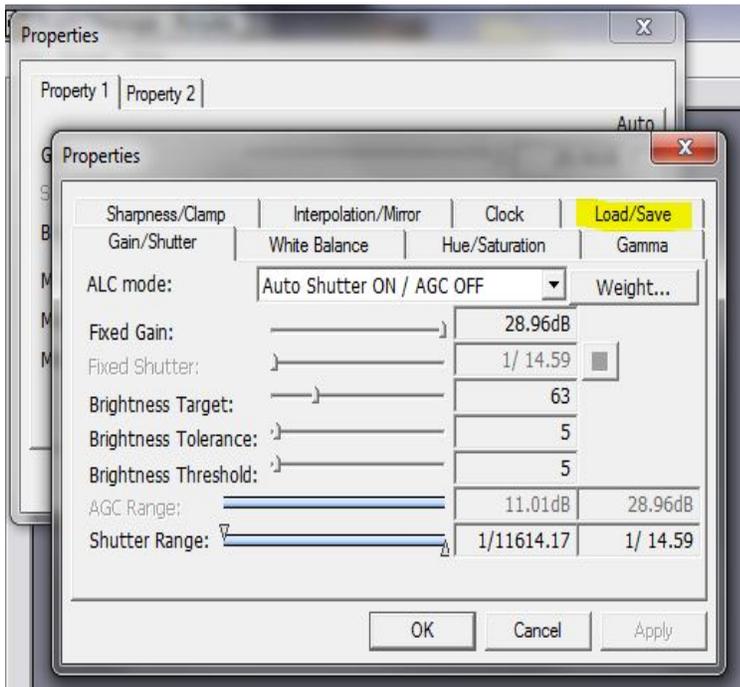


3. Click on **Load/Save**

4. Select your light source, either LED or SLS (LED shown highlighted), then click **Load**

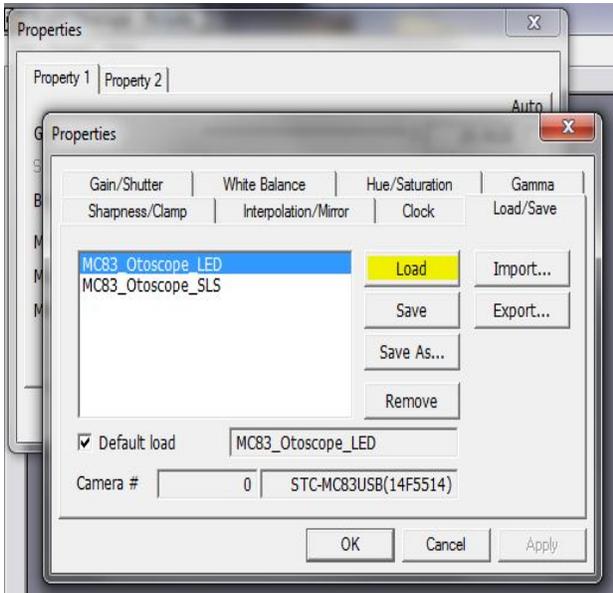


5. Click **Apply**, OK

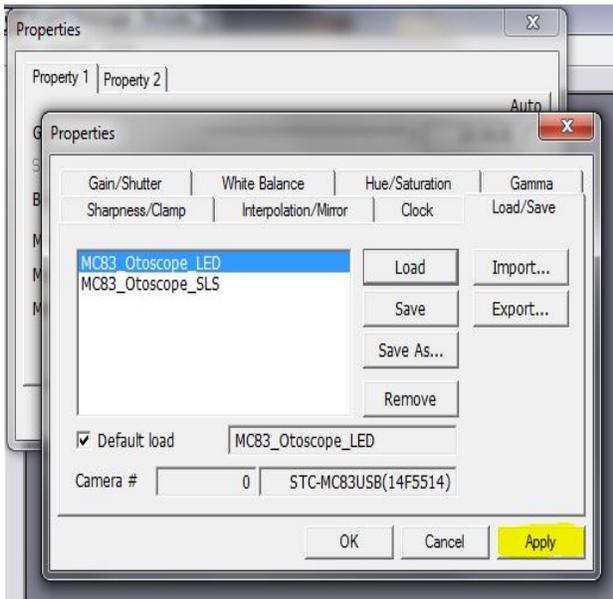

6. Click **Apply** in the Properties window, and then **OK** to complete Changes.


7. The setup is now complete

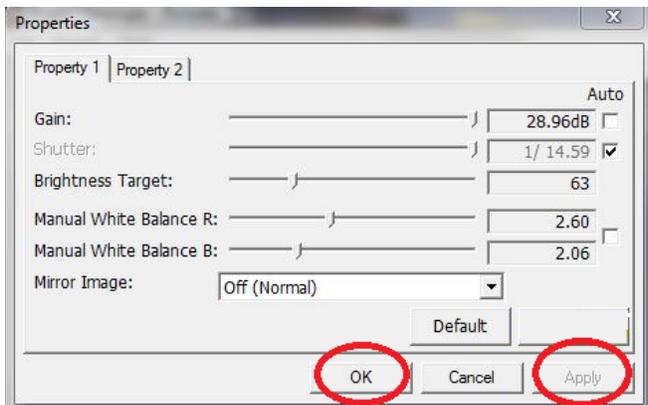

Taking Pictures

Within NOAH™ or stand-alone

*Always Use a clean speculum tip for each new patient.
Refer to "Set-Up Manual" for additional cleaning information.*

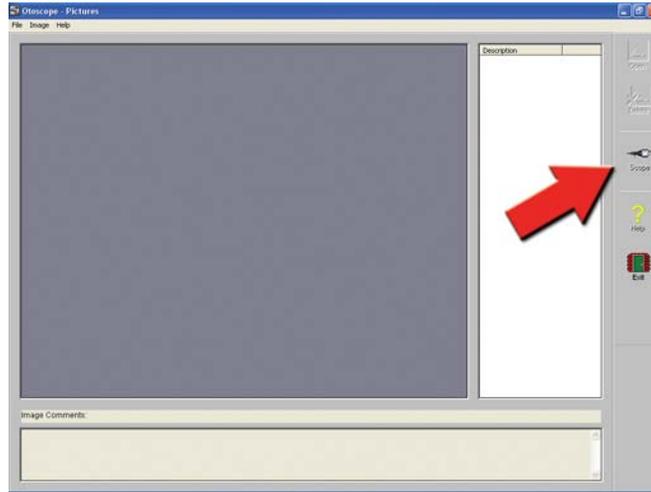

Click on the **Scope Icon** on the **Main Screen**.



The **Capture Screen** Appears.

The Capture Screen

Within NOAH™ or stand-alone



There are three ways to take pictures:

1. If you do not check either box:

When no items are selected, **click on the Capture Icon to take one picture**, the Main Screen appears.

2. If you check Single Click Capture.

When you click the Capture Icon instead of immediately taking a picture, the program switches to the Single Click mode.

All the icons will turn gray on the screen, indicating that they are inactive.

You can now take a picture by pressing any key on the keyboard, clicking the left mouse button or pressing the foot pedal, the Main Screen appears.

3. Or if you check both boxes.

When you click the Capture Icon instead of immediately taking a picture all the icons will turn gray, indicating that they are inactive.

You can now take as many pictures as you would like in a row without returning to the Main Screen **by pressing any key on the keyboard, clicking the left mouse button or pressing the foot pedal.**

To end the Single Click/Sequence mode, press the Esc key or the right mouse button.

Press the Back button to return to the Main Screen to work with your pictures.

Naming your pictures:

Use the pull-down menu at the top of your screen. Select from the list provided or highlight box and type in any name.


The Main Screen

Within NOAH™ or stand-alone



The Main Screen shows a list and preview of the captured images.

An image in the list can be renamed by selecting it, clicking the right mouse button, and choosing the Rename option from the menu.

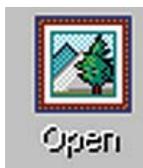

Open Icon:

Opens the selected picture in the **View Screen** where you can enhance it.



Scope Icon:

Click to take a new picture.

The View Screen

Within NOAH™ or stand-alone



This screen allows you to enhance your pictures.

Tool Bar



Use this button to enlarge the image.



Use this button to restore the image to the original size.



Use this button to maximize the image so that it occupies all available screen space.



Use this button to undo the last operation performed on the image.

Enhancing



Use this function to emphasize the details on the image.



Use this function to blur the details on the image.



Use this function to create a shadow effect on the image. Best when used with monochrome pictures.




Use this function to reduce motion artifacts.

Saving Images



Using NOAH: When you exit the program you will get a box that asks “do you want to Save NOAH session?” Click **Yes**.

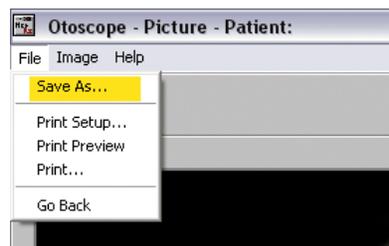

Not using NOAH: from the View Screen go up to the File Menu and choose Save As. Save your images to your desktop in jpg or bmp formats.



MedRx

www.medrx-usa.com



1200 Starkey Rd., #105, Largo FL 33771 U.S.A.

Toll Free: (888) 392-1234 • (727) 584-9600

Fax: (727) 584-9602 • Email: medrx@medrx-usa.com