

## STUDIO SOFTWARE QUICK KEYS

### GENERAL

ACTION	KEY	COMMENTS
Help	F1	Gives context help
Exit program	Alt-F4	Exits the program. Will prompt to save data
Go back to previous screen	Ctrl-Back	Same as clicking Back button
Scroll plot	Ctrl-Up / Ctrl-Down or Mouse Wheel	Only on plots with scrolling available
Toggle Ears	F2	
Change ear to Left	Ctrl-L	
Change ear to Right	Ctrl-R	
Switch Single/Dual Plots	Ctrl-1 for single, Ctrl-2 for dual, Ctrl-~ to toggle	Works only on screens that support dual plot mode.
Print	Ctrl-P	Print with template selection
Quick Print	Ctrl-Shift-P	Print with default template

### VIDEO QUICK KEYS

ACTION	KEY	COMMENTS
Capture Picture	Enter or Space or F2 or F12	Only for live video. Captures for currently selected side
Capture Left Side Picture	F8	Captures Left Side picture
Capture Right Side Picture	F9	Captures Right Side picture
Run/Freeze	F3 or F11	Only for live video
Save as Picture File	Ctrl-S	Save active picture as file. JPEG, BMP, PNG, GIF, and some other formats are supported
Open Picture File	Ctrl-O	Open a picture file
Copy to Clipboard	Ctrl-C	Copy active image to Windows Clipboard
Paste from Clipboard	Ctrl-V	Paste picture from Windows Clipboard

## AUDIOMETRY QUICK KEYS

ACTION	KEY	COMMENTS
Start signal	Space or Enter or S	Same as clicking on the Start button
Stop signal	ESC	
Start signal with the mouse	Point the cursor at the desired frequency/level points, push and hold Left mouse button, and then push the Right mouse button	
Increase level	ArrowUp or W or + (plus)	In Tone Audiometry, holding the Up or Down arrows will start a signal if AutoSound is selected in the Audiometry Options. Arrow keys can be reversed in the Audiometry options. Hold the Shift key to move levels by the Level Unit (set in options)
Decrease level	Arrow Down or Z or - (minus)	
Increase masking level	Ctrl-Arrow Up or Ctrl-W or Numpad2 or Ctrl + Numpad + (plus)	For dual-channel tests (Stenger, WR) controls Right channel. Hold Shift key to move levels by the Level Unit (set in the options)
Decrease masking level	Ctrl-Arrow Down or Ctrl-Z or Num pad8 or Ctrl + [Numpad - (minus)]	
Move frequency left	Left Arrow or A	
Move frequency right	Right Arrow or D	
Masking on/off	M	
Toggle channel lock	K	
No response	N	For tone audiometry only
Talk Forward	T or F12	
Monitors	O	
Toggle Phonemes View	P	
Toggle Hearing Loss Level View	I	
Toggle Banana Curve View	B	
Left ear	L	
Right ear	R	
Toggle ears	G	

Switch Signal Type	Shift-F1..F3	Only where Tone, Warble, NBN is available
Score Tone Point	Insert or H	Can be used if Auto-Record Tone Points is turned off
Delete Tone Point	Delete or Mouse Right Click	
Delete Curves	Ctrl + Delete	

## SPEECH AUDIOMETRY

ACTION	KEY	COMMENTS
Increase level	Arrow Right or Arrow Up or W or + (plus)	
Decrease level	Arrow Left or Arrow Down or Z or - (minus)	
Increase masking level	Ctrl-Up or Ctrl-W	
Decrease masking level	Ctrl-Down or Ctrl-Z	
Toggle channel lock	K	
Correct answer	F7	
Incorrect answer	F8	
Play word	F9	For Word Lists
Play next word	F10	For Word Lists
Talk Forward	T or F12	
Monitors	O	
Left ear	L	
Right ear	R	
Toggle ears	G	

## QUICK SIN

ACTION	KEY	COMMENTS
Check QuickSin Word 1	1	
Check QuickSin Word 2	2	
Check QuickSin Word 3	3	
Check QuickSin Word 4	4	
Check QuickSin Word 5	5	