



(888) 392-1234

www.medrx-diagnostics.com

Probe Tube Insertion Guide

*for Live Speech Mapping
and Conventional REM*

*(re: Pomford, J & Sinclair, S. Real-Ear Measurement:
Basic Terminology and Procedures. Audiology OnLine.com)*



Adult Male (30mm)



Adult Female (28mm)



Child (20-25mm)

1. Perform Otoscopic Examination
2. Align end of Probe Tube with the left end of the appropriate guide
3. Align tube marker with the red marker above
4. Place Probe Tube in ear canal
5. Confirm placement with Otoscope
6. Perform Live Speech Mapping & REM