



Loading Calibration for the AVANT ARC, REM or REM⁺ Unit

The calibration values for the probe microphone(s) must be entered into your AVANT REM Software manually.

Open the AVANT REM software

- Click the **FUNCTION** tab
- Click the **CALIBRATE** tab
- Click the **LOAD CALIBRATION FILE** tab
- Click the **MANUAL INPUT** tab
- Select **LEFT** (blue) **or RIGHT** (red) **probe tab**.

- **Enter the values on the calibration sticker, attached to the probe microphone cable, into the Reference and Probe microphone dB offsets.**

*If the value is a negative figure, you must enter the minus sign (-).
If the value is a decimal number, you must enter the period (.)*

- Click **SET**
- If the values displayed are correct, click the **YES** tab.

When you are finished entering left, right or both probe microphone dB offsets, click **CLOSE**.

MedRx[®]

www.medrx-usa.com • 1200 Starkey Rd., #105, Largo FL 33771 • 727-584-9600 • Toll Free 888-392-1234
