


Apple Computer MFi Connectivity

With the latest update to macOS® Sonoma, Apple® computers* can now connect to MFi hearing aids.

Please follow these steps to connect hearing aids to the Apple computer:

1. Click on the Apple icon in the top left corner of the screen and select **System Settings**. From the **System Settings** screen, click on **Accessibility** in the left-hand side of the screen. A new list will appear in the main System Settings screen. Select **Hearing Devices**.

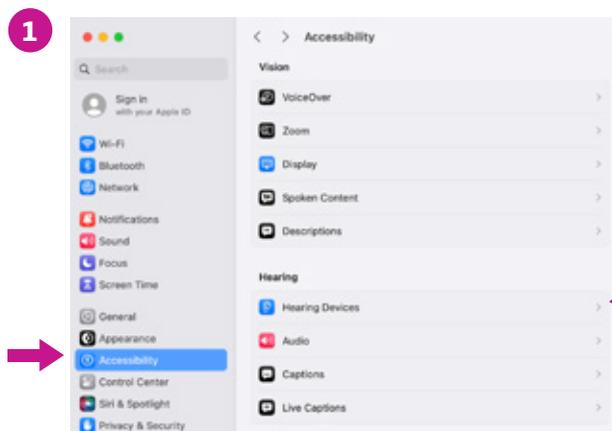

2. At this point restart the hearing aids to put them into pairing mode. The devices will appear on the main screen. Click the **Connect** button. We recommend turning off Bluetooth® on your other Apple devices when initially connecting to your Apple computer to ensure a quick connection.



3. You are now paired with your Apple computer and can change volume and programs, mute the aids and stream audio.



A shortcut can be made for users to have quick and easy access to make volume and program changes to their hearing aids from their desktop. Simply go to **System Settings > Control Center** and on the main menu find **Hearing** and turn on **Show in Menu Bar**. This will provide a shortcut denoted by an ear symbol in the menu bar, which is located at the top of your desktop's screen.

*Streaming is compatible with computers using macOS Sonoma 14 or later and MacBook Pro® introduced in 2021 or later, MacBook Air® introduced in 2022 or later, Mac Studio® (all models), Mac Pro® introduced in 2023 or later.