

Give your patients the power to go

The easy-to-use, portable Oticon SmartCharger gives patients freedom to enjoy life on the go. It features a built-in power bank and a lid for additional protection. What's more, it helps to keep hearing aids working well by automatically drying any moisture build-up inside of them. The SmartCharger offers convenient, reliable charging without access to a power outlet.

Patients now have the option to purchase the sleek, portable SmartCharger to give them the freedom to go wherever life takes them. The SmartCharger keeps rechargeable Oticon More™, Oticon Zircon, Oticon Play PX and CROS PX powered up, dry and protected.

Compatible with:
Oticon More
Oticon Zircon
Oticon Play PX
Oticon CROS PX

Oticon
SmartCharger*



Oticon SmartCharger:

- Recharges hearing aids without access to a power outlet—minimum of three full charges
- Designed for travel with a protective lid and built-in power bank
- Automatically removes moisture from hearing aids
- Charges hearing aids in **three hours**** for a full day of power†
- **30-minute quick charge** provides an additional six hours of power***



* Please note, miniBTE R uses a different SmartCharger and desktop charger than miniRITE R.

** For miniRITE R. Three and a half hours gives a full day of power for miniBTE R.

*** For miniRITE R. 30 minutes gives an additional five hours for miniBTE R.

† Lithium-ion battery performance varies depending upon hearing loss, lifestyle and streaming behavior.

Charging times and light status



Charging times

Hearing aids and SmartCharger power bank can charge simultaneously.



Oticon Hearing Aids		
miniRITE R		
3 hours	1 hour	0.5 hours
Fully charged	50% charged	25% charged
miniBTE R		
3.5 hours	1 hour	0.5 hours
Fully charged	44% charged	22% charged

Oticon SmartCharger	
4 hours	Up to 8 hours
Fully charged power bank battery	Fully charged power bank and hearing aids

Charging the SmartCharger

When the charger is connected to a power source, the LEDs on the back of the charger indicate the charging status of the power bank.

🔴	Charging (battery too low to recharge the hearing aid)
🟠	Charging
🟡	
🟢	
🟢	Fully charged

🔴 ORANGE blink 🟠 ORANGE 🟢 GREEN

Please Note: Oticon More instruments will need firmware version 1.1.1 or later in order to work with the SmartCharger.

Power bank function

When the charger is disconnected from power, the LEDs on the back of the charger indicate the charger's power bank battery level. Your patient can check the remaining battery level by opening the SmartCharger's lid.

🟢🟢🟢	Three full charging cycles of two hearing aids
🟢🟢	Two full charging cycles of two hearing aids
🟢	One full charging cycle of two hearing aids
🔴	Battery too low for a full charging cycle

🔴 ORANGE blink 🟢 GREEN