INSTRUCTIONS FOR USE

Zpower[®]

RECHARGEABLE System for Hearing Aids

Limited Warranty

ZPower's exclusive limited warranty is available at www.zpowerhearing.com. You may also obtain a copy of ZPower's Limited Warranty from your hearing care provider.

Intended Use

The intended use of the ZPower Rechargeable System is to convert your hearing aids into rechargeable hearing aids compatible with ZPower silver-zinc rechargeable batteries.

Compatibility

The ZPower Rechargeable System is available in multiple models, each designed for a specific set of hearing aid models. For a complete list of compatible models, please visit www.zpowerhearing.com.

Explanation of Symbols

Symbol	Explanation
	It is important to read the user manual.
	Points out situations that could lead to serious, moderate, or minor injuries or could damage your system.
	Advice and tips on how your device should be properly used and how to avoid damaging your system.
	With the CE symbol, ZPower confirms that this system and its individual components meet the requirements of the Medical Device Directive 93/42/EEC, RoHS Directive 2002/95/EC, WEEE Directive 2012/19/EU, & Battery Directive 2006/66/EC. The numbers after the CE symbol correspond to the code of certified institution that were consulted on the above mentioned directives.

Explanation of Symbols

Within the European Union, the marked device is covered by WEEE Directive 2012/19/EU and by RoHS Directive 2011/65/EC. Please dispose of this device by taking it to your local collection point or recycling center for such equipment.

The marked device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

There is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

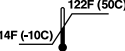
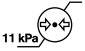
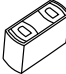
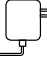
Indicates the medical device manufacturer, as defined in EU Directive 93/42/EEC.

Indicates the manufacturer's catalogue number so that the medical device can be identified.

Explanation of Symbols

 The symbol consists of the word "LOT" in a bold, sans-serif font, enclosed within a rectangular border.	Indicates the manufacturer's batch code so that the batch or lot can be identified.
 The symbol shows a thermometer with a vertical scale. The top of the scale is labeled "122F (50C)" and the bottom is labeled "14F (-10C)".	Indicates the temperature limits to which the medical device can be safely exposed: 14-122°F (-10-50°C)
 The symbol features a circular gauge with a percentage sign inside. A wavy line indicates a range from "20%" at the bottom to "75%" at the top.	Indicates the range of humidity to which the medical device can be safely exposed: 20-75%
 The symbol shows a circular gauge with a pressure value of "11 kPa" written below it. The gauge has a needle pointing to the right.	Indicates the range of atmospheric pressure to which the medical device can be safely exposed: minimum 11kPa
 The symbol is a simple line drawing of an umbrella with rain falling from it.	Indicates that the medical device needs to be protected from moisture.
 The symbol shows two cylindrical batteries, each with an "X" on its top surface.	Indicates that two Silver-Zinc batteries are included in the system.
 The symbol shows two square-shaped modules, each with a circular opening on one side.	Indicates that two Battery Door Modules are included in the system.
 The symbol is a line drawing of a rectangular device with a handle on top and a small rectangular opening on the front.	Indicates that a charger is included in the system.
 The symbol is a line drawing of a rectangular power adapter with a two-prong electrical plug on the right side and a cable extending from the bottom.	Indicates that an AC adapter is included in the system.

This User Guide is Valid For:

ZPower Rechargeable System including	
Battery Door Module (ZBDM-01)	2016
Charger (ZC-B01) & AC Adapter	2016
Silver-Zinc Rechargeable Batteries	2016

1 How to Use Your System

To get started, plug the power cord into the USB wall adapter and into a power outlet. When it's first plugged in, both charger lights will briefly turn green.

Charging Your Hearing Aid

1. Place Your Hearing Aids on the Charger

Lightly hold your hearing aids by the tube, and place them in the charging slot. Both domes should be facing the same side of the charger as the power cable, as shown in the picture. When your hearing aids are correctly placed, both lights will begin blinking green.

2. Wait 3-4 Hours

When the green lights are blinking, your hearing aids are charging. When the lights stop blinking and turn solid green, your hearing aids are completely charged.

3. Remove, And Wear Your Hearing Aids

You can remove your hearing aids before they're completely charged, but they won't last as long as when they're fully charged. It's best to leave your hearing aids in the charger anytime you're not using them, they won't overcharge.

 Please ensure to charge your hearing aids within the operating temperature range: 50-104°F (10-40°C).


Hearing Aids in Charger

Charger

Battery & Door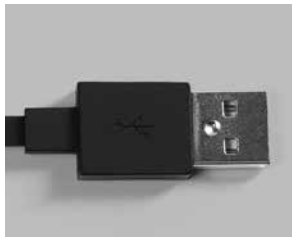

Charger USB Cable

USB Wall Adapter

2 Important Instructions

This charger is intended to charge hearing instruments with the ZPower Rechargeable System. Use only with ZPower rechargeable batteries. This product does not have a power switch. It is turned on once you plug in the power cord. The USB Wall Adapter will conform to the requirement of your region for voltage input and connection type.

i NOTICE

- This charger is intended only for indoor use.
- Do not expose your device to extreme temperature or high humidity.
- Do not leave it in direct sunlight.
- Do not immerse it in water.
- Do not place your device in a microwave oven.

The hearing care professional shall fit the rechargeable battery door module to appropriate model hearing aid. Attempting to install a battery door on an incompatible hearing aid may result in breaking into small and/or sharp pieces. Only install a battery door on compatible hearing aid models. For a list of compatible models, visit www.zpowerhearing.com.

 **WARNING**

- **Choking hazard.** Choking hazard posed by small parts. Keep hearing instruments, batteries and accessories out of the reach of children and pets. If any part of the system is swallowed, seek immediate medical attention.
- **Tripping hazard.** Keep long power cord out of walkway where it poses a tripping hazard.
- **Magnetic field.** Charger contains magnets and should be kept at least 6 inches (15 cm) away from pacemakers.
- **Strangulation hazard.** Contains a long power cord that may cause strangulation if used improperly.
- **Risk of injury.** Do not hold power cord and swing charger.
- **Electric shock.** Risk of electric shock through incompatible parts. Use the USB wall adapter provided with the charger.
- **Fire and explosion hazard.** Do not use in explosive or oxygen-enriched atmospheres.
- **Ingestion hazard.** If magnets are separated from charger, do not put in nose or mouth. Swallowed magnets can stick to intestines causing serious injury or death. Seek immediate medical attention if magnets are swallowed or inhaled.
- **Overheating hazard.** Use of an improper USB wall adapter may cause the charger to overheat. Use only EN60950 certified equipment with an output of 5 VDC, minimum 500 mA, such as the USB wall adapter provided with the charger.
- Use the USB port of the charger for the described purpose only.
- Do not insert the wall adapter alone into electrical outlets. This could result in electric shock.
- Use only the equipment described in these instructions to charge your system. Using other equipment may result in damage to your system.
- Do not drop any part of the rechargeable system onto a hard surface, this may damage your system and could result in breaking into small and/or sharp pieces

3 Maintenance

Proper use will help to extend the life of your battery.

Battery Tips


- Charge the batteries regularly before wearing them.
- Charge the batteries at temperatures between 50°F (10°C) and 104°F (40°C).
- Don't place the batteries in the charger if the charger is unplugged, this will drain the batteries.
- If you don't intend to use your hearing aids for several days, remove the batteries.
- Your battery should last for approximately one year, and can be easily replaced by your hearing professional at that time.

Cleaning


- Clean as needed.
- Use a dry, soft tissue to remove any debris.
- The soft brush that is included with hearing aids may be used to clean the contacts on the charger.
- Do not sterilize or disinfect the charger.
- Do not clean with rubbing alcohol, solvents or any other liquid.

i Your hearing aid and/or battery door may be damaged by hairspray or cosmetics. It is recommended to remove your hearing aids from your ear before applying these products.

! Always make sure that the hearing aids and charger are dry and clean before charging.

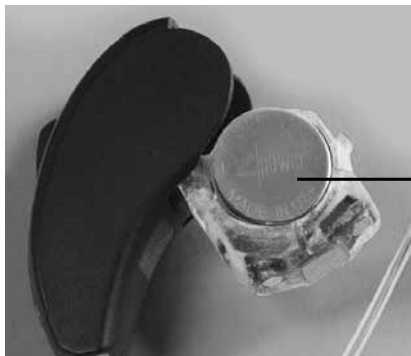○ **Make sure the rechargeable batteries are inserted correctly.**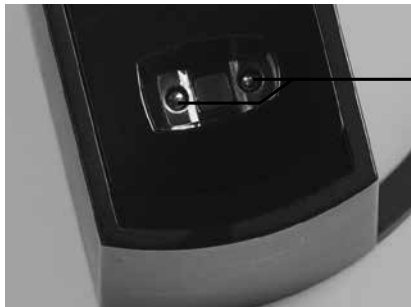

○ **Use a dry, soft tissue to clean the contacts on the charger. The soft brush that is included with hearing aids may be used to clean the contacts on the charger.**

4 Troubleshooting

Charger Lights	Problem	Recommended Action
No lights	Hearing aid is not inserted into the charger correctly or the charger is not powered.	Remove hearing aid(s) and replace in correct orientation. Verify USB cable is properly connected to USB wall adapter. Verify USB wall adapter is functional by plugging in another USB device.
One or both lights solid red	Battery won't charge.	Verify hearing aid battery is a ZPower rechargeable battery.
Both lights flashing red	Temperature is out of range. Charging is paused until temperature is in range.	Move charger to an environment between 50°F (10°C) and 104°F (40°C).
Consult your hearing care professional if any problem persists.		

5 Technical Data

Specifications for the ZPower Rechargeable System

Operating temperature	50°F (10°C) to 104°F (40°C)
Storage temperature	14°F (-10°C) to 122°F (50°C)
Humidity	20% to 75%
Protection against ingress of water	IPX0 (ordinary)
Minimum air pressure	11 kPa
USB wall adapter input voltage	100-240V AC
USB wall adapter output voltage	5V DC
USB wall adapter output current	<=1.0A
USB wall adapter:	ZPower provided CB certified wall adapter
Protection against electric shock	Class II

See www.zpowerhearing.com for hearing aid compatibility list.

6 Compliance Information

Hereby ZPower LLC declares that this product meets the requirements of the Medical Devices Directive 93/42/EEC. The full text of the Declaration of Conformity can be obtained from the manufacturer or the local representative whose address can found on the back cover of these instructions.

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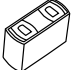
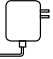
 The symbol consists of the word "LOT" in a bold, sans-serif font, enclosed within a rectangular border.	指示該製造商的批碼是可識別的。
 The symbol shows a thermometer with a vertical scale. The top of the scale is labeled "122F (50C)" and the bottom is labeled "14F (-10C)".	指示該醫療裝置可暴露於溫度安全範圍為：14-122°F (-10-50°C)
 The symbol features a circle containing a percentage sign (%). A wavy line at the bottom of the circle is labeled "20%" and a line pointing to the top of the circle is labeled "75%".	指示該醫療裝置暴露於濕度安全範圍為：20-75%
 The symbol shows a circle with a double-headed arrow pointing horizontally across it. Below the circle, the text "11 kPa" is written.	指示該醫療裝置可暴露於大氣壓力的安全範圍為：最小11kPa
 A simple line drawing of an open umbrella with raindrops falling from it.	指示該醫療裝置需要保護免受濕氣。
 The symbol depicts two cylindrical zinc-carbon batteries, one slightly behind the other.	指示該兩個銀鋅電池是包括在系統中。
 The symbol shows two circular battery door components, each with a small tab on its side.	指示該兩個電池門的組件是包括在系統中。
 The symbol is a line drawing of a rectangular electronic device, likely a charger, with two small rectangular ports on its top surface.	指示該充電器是包括在系統中。
 The symbol shows a rectangular AC adapter with a power cord extending from its bottom and a two-prong electrical plug on its right side.	表示AC適配器是包括在系統。

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ZPower, LLC
U.S.A.