

Getting Started with Amigo Star and T5 FM Transmitter



Get to know the Amigo T5

- | | |
|----------------------------|----------------------------|
| 1 ON/ OFF | 6 LED |
| 2 Channel display | 7 Mute button |
| 3 Channel up | 8 Charging plug |
| 4 Channel down | 9 Aux input |
| 5 Microphone (boom) | 10 Microphone input |

Getting to know the Amigo Star

- | |
|-------------------------------|
| 1 Volume control wheel |
| 2 LED indicator |
| 3 Channel seek button |



Using Amigo Star with T5 transmitter



The Amigo T5 is a body worn FM transmitter.

- 1 Insert 2 AAA batteries into T5 and turn the transmitter on. The red LED will turn on.



- 2 Insert a battery into Amigo Star, close the battery door, and roll the volume control past position 1.

- 3 **Press the channel seek button**
The channel seek button is used for finding the channel that the Amigo T5 is on. Once the channel is found, the LED light on Amigo Star will light up, to indicate the system is now working.

Connect the Corda² tube and dome to Amigo Star



- 1 **Select the appropriate Corda² tube.**
The size of the tube is printed in either blue numerals (left) or red numerals (right) at the end of the tube.


Most children will use the smallest tube labelled 0-0.9.

Attach the tube to Amigo Star.



- 2 **Select the appropriate Corda² dome** and attach this to the tube. Most children will use the smallest dome.


Place the Amigo Star on the ear

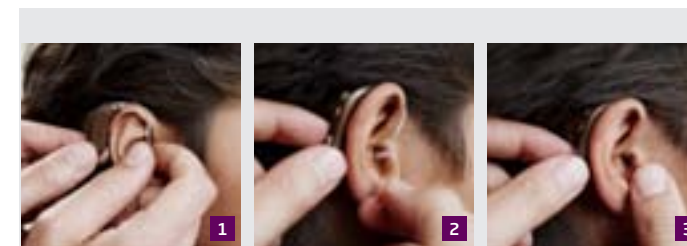

After the Amigo Star has the appropriate size Corda² tube and dome attached, and it has been programmed, it is ready to be placed on the ear.

- 1 **Place the Amigo Star over the ear**
- 2 **Position the dome in the ear canal and fold the ear grip into the ear**
- 3 **Gently push the Corda² thin tube in the ear**

Wearing the T5 transmitter

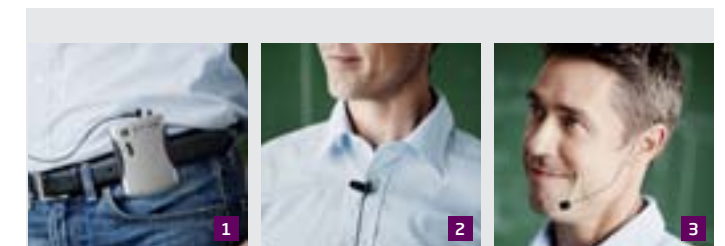

- 1 **Secure the T5 transmitter using the belt clip**
- 2 **Clip the lapel microphone onto clothing**
The microphone should be positioned 4 - 6 inches from the mouth.
- 3 **Using the boom microphone**
The boom microphone can be used as an alternative to the lapel microphone. Place the boom microphone over the ear and shape the wire so the microphone is positioned 0.5 - 2 inches from the mouth.

Checklist

No power/ no LED light on Amigo Transmitter

LED light on Amigo T5 flashing green

Student reports no sound

Student reports static sound

Action

Check, change or charge battery

Note: Amigo T5 can run on 2 regular AAA batteries

Indicates low batteries - charge Amigo or insert new AAA batteries

Check microphone connection

Is Amigo T5 muted? (Is the red LED flashing?)

Is Amigo T5 turned on? (Is the LED constant red?)

Check the LED on Amigo Star is flashing or constant red

Clean or replace the Corda² thin tube and dome on Amigo Star

Check Amigo Star is on the same channel as the T5

Press the channel seek button on Amigo Star to find channel

Check microphone cord is straight, free of kinks and knots

Check or change channel with channel up and down buttons

Additional functions can be accessed on the Amigo Star. These include locking the volume control wheel, push button and LED light.

To access these, use the WRP wireless programmer, or the T30/T31 transmitter.