

Amigo User Guide for Teachers

Amigo T30/T31 FM Transmitters



Getting started

- 1** CHARGE the Amigo with mini USB charger
- 2** Turn the Amigo ON. The constant LED will confirm ON status. This button is also the OFF button
- 3** Plug in the microphone
- 4** Clip on the microphone and begin talking
- 5** To MUTE, quickly press the ON button. The LED will flash to confirm MUTE status
- 6** AUXILIARY inputs can be plugged into Amigo. E.g. TV, Music player, Computer



Checklist

No power/ no LED light on Amigo

LED light on Amigo flashing green

Student reports no sound

Student reports static sound

Action

Check, change or charge battery

Note: Amigo can run on a regular AA battery

Indicates low battery - charge Amigo or insert new AA battery

Check microphone connection

Check audio shoe connection on hearing aid

Is Amigo muted? (Is the red LED flashing?)

Is Amigo turned on (Is the LED constant red?)

If the receiver has an LED, check it is flashing or constant red

Check microphone cord is straight, free of kinks and knots

Check or change channel with up and down arrow button


Transmitter and Microphone Placement



Transmitter with belt clip



Lapel microphone

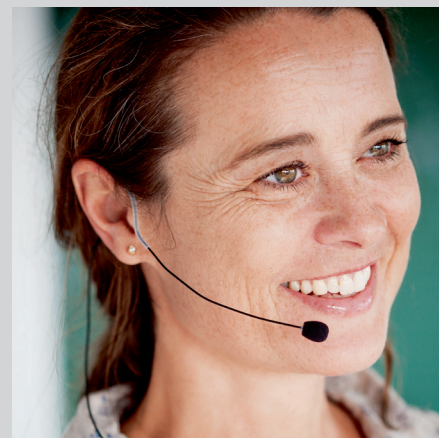

Boom microphone