

## Getting Started with Amigo Star and T30/T31 FM Transmitters



### Get to know the Amigo T30/T31

- |                               |                             |
|-------------------------------|-----------------------------|
| <b>1</b> LED Light            | <b>6</b> Navigation buttons |
| <b>2</b> ON/ OFF/ MUTE button | <b>7</b> Microphone (lapel) |
| <b>3</b> Microphone input     | <b>8</b> Charging port      |
| <b>4</b> C (Clear) button     | <b>9</b> Auxiliary port     |
| <b>5</b> Sync button/enter    |                             |

### Getting to know the Amigo Star

- |                               |
|-------------------------------|
| <b>1</b> Volume control wheel |
| <b>2</b> LED indicator        |
| <b>3</b> Channel seek button  |

# Programming with T30/T31 transmitter




The Amigo T30 and T31 are body worn FM transmitters. They are also the Amigo Star's programming unit. No other equipment is required.

Insert a battery into T30/T31 and turn the transmitter on.

- 1 Accessing the Programming Menus**  
Press the C button and the ON button simultaneously to access the Programming Menus.

### There are 3 main menus accessed from the Fitting Menu.

- View/ Program Transmitter
- View/ Program Receiver
- FM Channel Wizard


Press ▲▼ for navigation. Press  to make a selection. Press ◀ to return to the main menu screen. To exit the Programming Menus, press the C and ON button simultaneously.



- 2 Set channels in transmitter:**  
Go to the FM Channel Wizard menu. Press ▶ to Scan & Select Channel Set. All available channels will be tested and evaluated in order to find the best choice of channel. Press ◀ to exit and save the new channel set.



- 3 Program the Amigo Star:**  
Hold the Amigo Star and transmitter as shown. The orientation is important for communication between the devices.  
  
Go to the View/ Program Receiver menu.

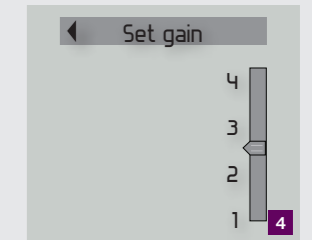


- 4 Disable the volume control wheel (if desired)**  
For young children or children with learning difficulties, the volume control wheel can be disabled.  
  
If the volume wheel is disabled, set the gain with the transmitter. This is the gain that Amigo Star will always be on.  
  
The numbers on the volume wheel correspond with the numbers on the display.

## Connect the Corda<sup>2</sup> tube and dome to Amigo Star




- 1 Select the appropriate Corda<sup>2</sup> tube.**  
The size of the tube is printed in either blue numerals (left) or red numerals (right) at the end of the tube.  
  
Most children will use the smallest tube labelled 0-0.9.  
  
Attach the tube to Amigo Star.
- 2 Select the appropriate Corda<sup>2</sup> dome**  
and attach this to the tube. Most children will use the smallest dome.
- 3 Insert a battery into Amigo Star,**  
close the battery door, and roll the volume control past position 1.

## Place the Amigo Star on the ear

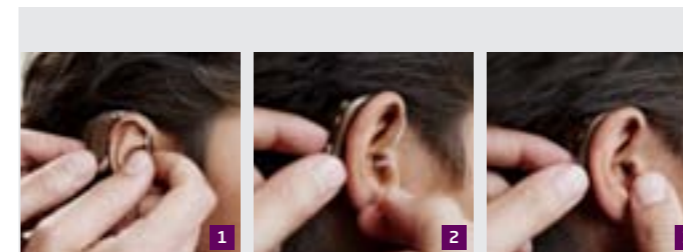


- After the Amigo Star has the appropriate size Corda<sup>2</sup> tube and dome attached, and it has been programmed, it is ready to be placed on the ear.
- 1 Place the Amigo Star over the ear**
  - 2 Position the dome in the ear canal and fold the ear grip into the ear**
  - 3 Gently push the Corda<sup>2</sup> thin tube in the ear**

## Wearing the T30/T31 transmitter

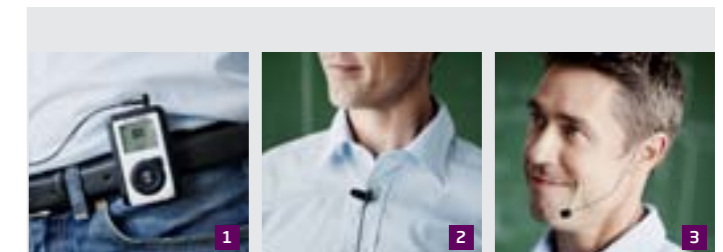

- 1 Secure the T30/T31 transmitter using the belt clip**
- 2 Clip the lapel microphone onto clothing**  
The microphone should be positioned 4 - 6 inches from the mouth.
- 3 Using the boom microphone**  
The boom microphone can be used as an alternative to the lapel microphone. Place the boom microphone over the ear and shape the wire so the microphone is positioned 0.5 - 2 inches from the mouth.

## Using the system

If the LED is not illuminated, press  to sync. The LED will light up, indicating Amigo Star is receiving speech from the transmitter. The system is now ready to use.

## Checklist

No power/ no LED light on Amigo Transmitter

Receiver out of range message appears

LED light on Amigo T30/T31 flashing green

Student reports no sound

Student reports static sound

## Action

Check, change or charge battery

Note: Amigo T30/T31 can run on a regular AA battery

Ensure Amigo Star is 1 - 15 inches from transmitter during programming

Hold Amigo Star and transmitter in the orientation shown on page 3

Check, change or charge battery in Amigo T30/T31

Check or change battery in Amigo Star

Indicates low battery - charge Amigo or insert new AA battery

Check microphone connection

Is Amigo T30/T31 muted? (Is the red LED flashing?)

Is Amigo T30/T31 turned on? (Is the LED constant red?)

Check the LED on Amigo Star is flashing or constant red

Clean or replace the Corda<sup>2</sup> thin tube and dome on Amigo Star

Check Amigo Star is on the same channel as the T30/T31

Press the  button to sync

Check microphone cord on T31 is straight, free of kinks and knots

Check or change channel with up and down button