

















# Oticon Amigo<sup>1</sup> and Oticon hearing aids

## DAI/FM overview

Hearing aids	DAI adapter		FM adapter	FM receiver <sup>1</sup>	Modes of operation
<p>Automatically adds 2 programs to the hearing aid:</p> <ul style="list-style-type: none"> <li>- DAI/FM + HA (default)</li> <li>- DAI/FM only</li> </ul>	<p>Low Level Input bodyworn FM solutions e.g. Amigo R5</p>	<p>High Level Input TV, stereo, MP3 player, CD player, computer</p>	<p>Adapter for Amigo receivers and other ear-level receivers</p>	<p>Universal and dedicated Amigo receivers</p>	<p>Default modes of operation when using DAI adapters and FM receivers</p>
 <p>Oticon Opn S™, BTE PP Oticon Opn Play™, BTE PP Oticon Opn™, BTE PP (FW 6.0) Oticon Xceed, BTE SP &amp; BTE UP Oticon Xceed Play, BTE SP &amp; BTE UP Oticon Ruby, BTE PP &amp; BTE Oticon Siya, BTE PP &amp; BTE Oticon Alta2/Nera2/Ria2, BTE PP*</p>	<p>AP 1000 142207 Gray</p> 	<p>EduMic 191009 White</p> 	<p>Amigo R2</p> 	<p>Amigo R2 switch positions: ○ or ○○ = HA + FM receiver ● = HA only/FM off</p>	
		<p>FM 10 142328 Transparent</p> 	<p>Amigo R2</p> 	<p>Amigo R2 switch positions: ○ or ○○ = HA + FM receiver ● = HA only/FM off</p>	
			<p>Amigo R12G2</p> 		
			<p>Amigo Arc</p> 	<p>Via telecoil</p>	
 <p>Oticon Alta2/Nera2/Ria2, RITE &amp; BTE Oticon Dynamo, BTE SP Oticon Sensei, RITE &amp; BTE Oticon Sensei SP, BTE SP Oticon Get**, BTE &amp; BTE Power (Style exceptions: designRITE, miniRITE, BTE PP and miniBTE)</p>	<p>AP 900 399-50-520-08 Gray</p> 	<p>Streamer Pro 151240 White 151237 Black</p> 	<p>Amigo R2</p> 	<p>Amigo R2 switch positions: ○ or ○○ = HA + FM receiver ● = HA only/FM off</p>	
		<p>FM 9 399-50-590-00 Transparent</p> 	<p>Amigo R2***</p> 	<p>Amigo R2 switch positions: ○ or ○○ = HA + FM receiver ● = HA only/FM off</p>	
			<p>Amigo R12</p> 		
			<p>Amigo Arc</p> 	<p>Via telecoil</p>	

<sup>1</sup>Dedicated Amigo receivers are no longer commercially available.

\* EduMic is not compatible with Oticon Alta2/Nera2/Ria2.

\*\* Streamer Pro is not compatible with Oticon Get family.

\*\*\* On 312 battery style, the LED on Amigo R2 must be set to blink.

# Order information

## DAI Cords

Cord, Monaural	CD or MP3 Players, computer, etc	020100-794
Cord, Binaural	CD or MP3 Players, computer, etc.	383-58-941-00
DAI Y cord	60 cm (24") white	384-01-066-00
DAI Y Cord	60 cm (24") gray	384-01-063-00
DAI Y Cord	75 cm (30") white	384-01-070-00
DAI Y Cord	75 cm (30") gray	384-01-068-00
DAI Cord	60 cm (24") white	384-01-065-00
DAI Cord	60 cm (24") gray	384-01-062-00
DAI Cord	75 cm (30") white	384-01-069-00
DAI Cord	75 cm (30") gray	384-01-067-00