

## Oticon ConnectLine Digital Office Phone Kit Easy Setup Instructions

These instructions guide you through setting up ConnectLine's digital office phone solution. There are two methods for installation:

1. Hard-wired setup
2. Wireless setup

### 1. HARD-WIRED SETUP METHOD

You will use Streamer Pro in conjunction with the Sennheiser box, coiled phone cable, and AUX cable, pictured below:

### Setup Instructions

(Before beginning, insert the included battery into the Sennheiser device.)



Sennheiser box & coiled phone cable



Insert coiled cable into the back port of the Sennheiser box as pictured.



Remove phone receiver cord from phone base & replace with coiled cable from Sennheiser box. The coiled phone cable will now connect the Sennheiser box and phone base.



AUX cable



Insert phone receiver cord into front of Sennheiser box.



Insert the AUX cable into the headset jack on the Sennheiser device as pictured. Plug the mini jack end into Streamer Pro.\*

### Handling calls using the hard-wired setup method:

- No alerts will be heard through the hearing instruments. (the ring sounds on the phone only)
- Make sure the AUX input is selected on Streamer Pro. Press the AUX button if necessary. (note: not the phone button)
- Ensure the phone button on the Sennheiser box is in the up position. (not pressed down)
- When a call comes through, pick up the phone receiver and place it on the desk or table surface. Conduct the conversation through the Streamer Pro as usual. Replace the receiver when through.
- For outgoing calls, lift the receiver and place it on the desk or table surface. Dial the number and the call will go through. Replace the receiver when the call is ended.

### Control settings on the bottom of the Sennheiser box:

- Set the ABC switch to the position that provides the best sound quality.
- Rotate the small volume control dial to achieve the best balance of sound quality and volume for both parties.

\*Ensure Streamer Pro is around the user's neck

If your digital office phone has a headset port, you can also set up the digital office phone kit using this functionality. Instructions for this method are available in the Product Literature section at [www.oticonusa.com/connectline](http://www.oticonusa.com/connectline) in the booklet called "Digital Office Phone Kit Setup Guide."

# Digital Office Phone Kit Easy Setup Instructions

## 2. WIRELESS SETUP METHOD

You will use Streamer Pro in conjunction with the Phone Adapter 2.0 & power source, Sennheiser box, coiled phone cable, and AUX cable, pictured below:



Phone Adaptor 2.0 & power source



Sennheiser box & coiled phone cable



AUX cable

### Control settings on the bottom of the Sennheiser box:

- Set the ABC switch to the position that provides the best sound quality.
- Rotate the small volume control dial to achieve the best balance of sound quality and volume for both parties.

\*Ensure Streamer Pro is around the user's neck.

## Setup Instructions

(Before beginning, insert the included battery into the Sennheiser device.)



Connect the power supply and mini jack end of the AUX cable to the Phone Adapter 2.0 as pictured.



Follow steps 1-3 from the hard-wired setup instructions (opposite side). Then, connect the AUX cable to the Sennheiser box as pictured.



Ensure the Phone Adapter 2.0 is plugged in and has been paired with the Streamer Pro. You can now handle calls wirelessly with Streamer Pro.\*

### Handling calls using the wireless setup method:

- No alerts will be heard through the hearing instruments. (the ring sounds on the phone only)
- Ensure the phone button on the Sennheiser box is in the up position. (not pressed down)
- When the phone rings, press the Phone button on Streamer Pro first. Wait until you see a green and blue light on the Phone Adapter 2.0.
- Pick up the receiver and place it on the desk or table surface. Conduct the conversation through the Streamer Pro as usual. When finished, replace the receiver and push the Phone button in Streamer Pro to end the wireless connection.
- For outgoing calls, press the Phone button on Streamer Pro to establish the wireless connection. Pick up the receiver, set it down, and dial the number. The call will go through. When finished, replace the receiver and press the Phone button on Streamer Pro to end the wireless connection.