



USER GUIDE TV listening Daily use

INTRO

This TV solution is a wireless connection between your Streamer and the preferred TV in your home.

It gives you high quality TV sound in the hearing instruments within a range of 30 feet from the TV.

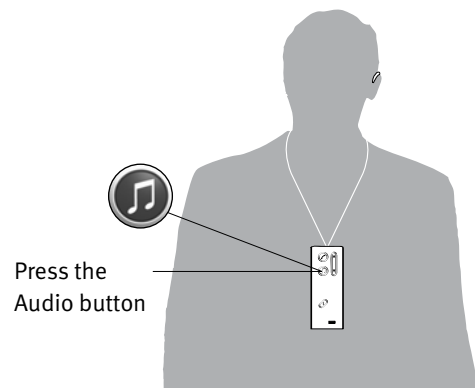
Before using this guide, make sure that the TV adapter is correctly connected to both the TV and power. Refer to the installation guide.

This user guide explains how Streamer is used for TV listening.

STREAMER OVERVIEW



When fully charged, Streamer can be used for approx. 5 hours of TV listening.

**Before listening**

Wear Streamer around your neck using the neck strap. You must be within **30 feet** from the TV set.

Use Streamers' Audio button for TV listening.

**START sound**

Press the Audio button for a short while. Streamer connects to the TV adapter. After a few seconds an orange solid light is shown on Streamer and TV adaptor. The hearing instruments will receive the TV sound.



STATUS

STOP sound

Press the Audio button for a short while. All orange lights turn off. Hearing instruments go back to standard program (P1).

Notice: The Connect function on the Streamer must be activated to allow transmission from the TV. If not active, it automatically turns on when starting the TV listening. De-activating the Connect function must be done manually.

Use the Volume button on Streamer to adjust the TV volume in the hearing instruments.

Adjust the volume by pressing Up or Down shortly. Repeat the **short press** to do further adjustments.

The TV volume set by Streamer is memorized. The latest setting will also be the used volume level applied next time TV listening is started.

Notice: The TV volume in the hearing instruments is normally not affected by the volume adjustments from the TV remote control.

**TV SOUND ONLY**

First time listening to TV, the hearing instrument microphones are default turned on.

The hearing instrument microphones can be muted allowing you to hear the TV sound only.

This is done by pressing the Volume button



2 sec

either Up or Down for **2 seconds**. Pressing the Volume button for 2 seconds again activates the hearing instrument microphones.

Notice: The microphone state is memorized. The latest setting will also be the microphone state next time TV listening is started.

For detailed user instructions on Streamer, refer to the Streamer manual.

Further relevant tips and support can be found on www.oticon.com/connectline