- EN ▶ HOW TO CHECK YOUR CHILD'S HEARING AID
- DK ▶ HVORDAN DU TJEKKER DIT BARN'S APPARATER
- SE ▶ SÅ HÅR KONTROLLERAR DU DITT BARNS HÖRAPPARATER
- NO ▶ HVORDAN SJEKKE BARNETS HØREAPPARAT
- NL ▶ ZO KUNT U HET HOORTOESTEL VAN UW KIND CONTROLEREN

HEARING AID
LISTENING TUBE
USER GUIDE

People First

People First is our promise to empower people to communicate freely, interact naturally and participate actively

child friendly hearing care

Our paediatric audiological mission is to ensure a better future for every child with hearing loss. We will deliver solutions, tools and techniques that optimise auditory and cognitive habilitation, embrace the complexities of growing up with hearing loss and empower you to adapt solutions to each child's developmental stage on their journey to adulthood.

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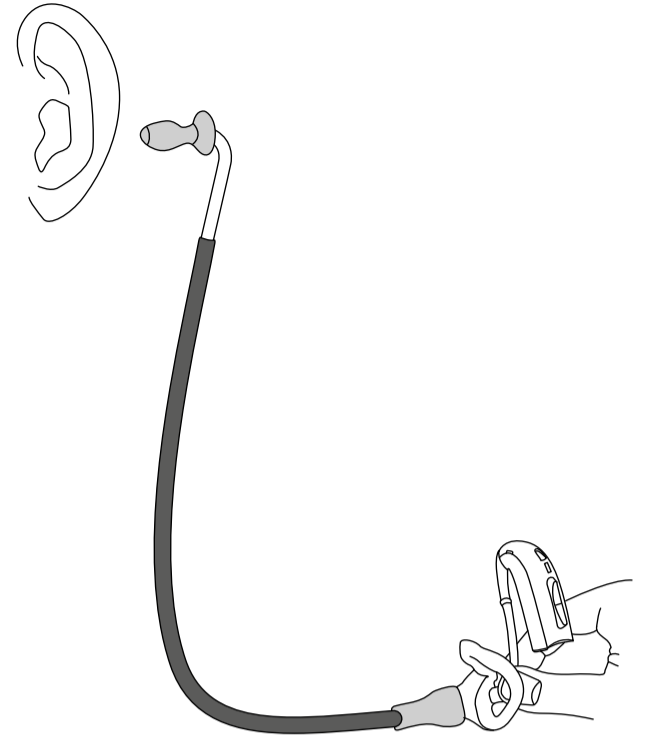
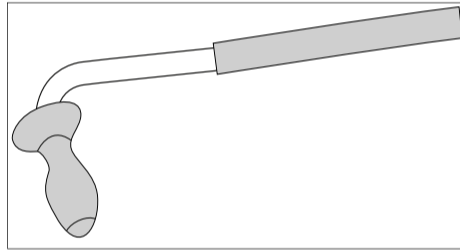
EN HOW TO PERFORM A LISTENING CHECK OF YOUR CHILD'S HEARING AIDS

We have provided a special listening tube, which allows you to listen to the hearing aids without putting them in your ear.

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Steps 3-4

3a

EN If you are checking a hearing aid that fits behind the ear, as shown, attach the other end of the listening tube to the earmould of the hearing aid.

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3b

EN If you are listening to a hearing aid with a dome or mould as shown, remove the dome and attach the listening tube directly to the sound outlet.

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4

EN Turn the hearing aid on by closing the battery door. You should hear a start-up jingle, unless the jingle has been turned off by the hearing care professional.

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Steps 5-7

5

EN With the ear piece of the Listening Tube in your ear, hold the hearing aid out in front of you, but not too close to your mouth. Listen for the clarity of these sounds as you say them aloud: "AH" as in father; "EE" as in bee; "OO" as in moon; "SH" as in shoe; "SSS" as in sun; and "MMM" as in mother.

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6

EN Checking the Telecoil (T-coil): A T-coil with a loop system broadcasts directly to the hearing aid when it is in the T-program. If your child uses a telecoil program you can also perform a listening check of this. Turn the hearing aid to the "MT" or "T" program.

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7

EN Hold the hearing aid near a television that is turned on. A buzzing sound will confirm that the telecoil is working.

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