

Oticon On IFTTT guide

Change hearing program when I enter a location

What you will need:

Opn hearing aid · iPhone 5 (or later) · Oticon ON app
ON account · IFTTT account · IF app

Initial Preparation

How to set up your iPhone:

- Go to app store on your iPhone
- Find the Oticon ON app in the apps section of the search results and click on the icon
- Find and press the Install button to download and install the app on your iPhone
- In the same way, install the 'IF' app.
- Open the Oticon ON app. Follow the instructions in the ON account creation section in the app

How to create an IFTTT account:

- Go to www.ifttt.com
- Press 'sign up' in the upper right corner
- Fill in your information and press 'Create account'

Create your free account

You're only seconds away from doing more with the products you love.


Your Email

Choose a Password

[Create account](#)

1

Go to www.ifttt.com, log in with your IFTTT account and click on 'My Recipes' in the top bar, followed by 'Create a Recipe'.

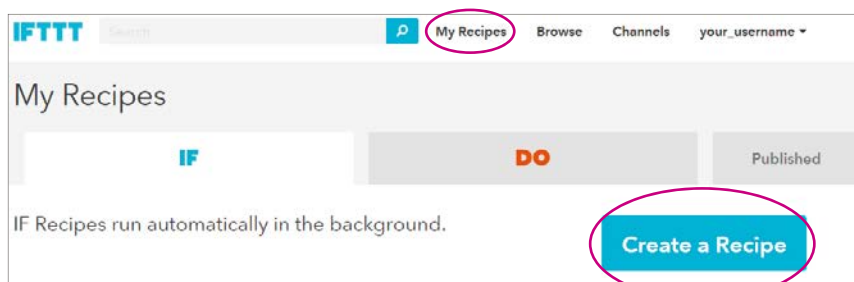


2

Click on the big blue 'this'




3

In the search field, type 'location', and click on the 'iOS Location' button

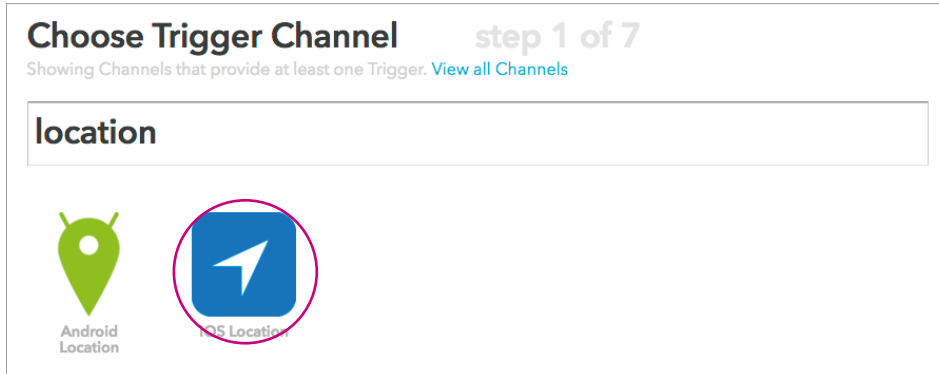


4

Choose when you want to switch to another program by clicking on the corresponding button

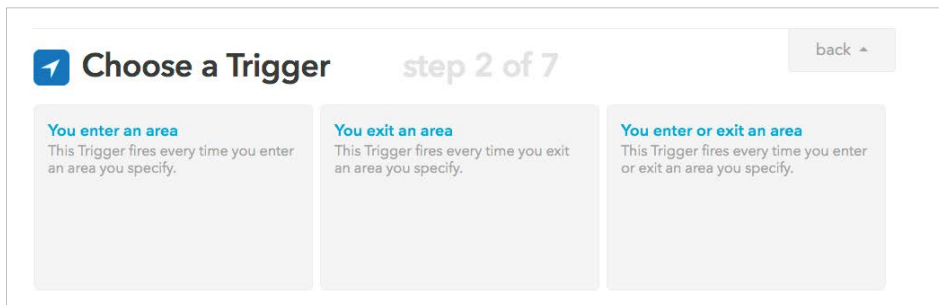

5

Mark your location by searching and highlighting the area you want to use in this recipe. When you finished, press on 'Create Trigger'.

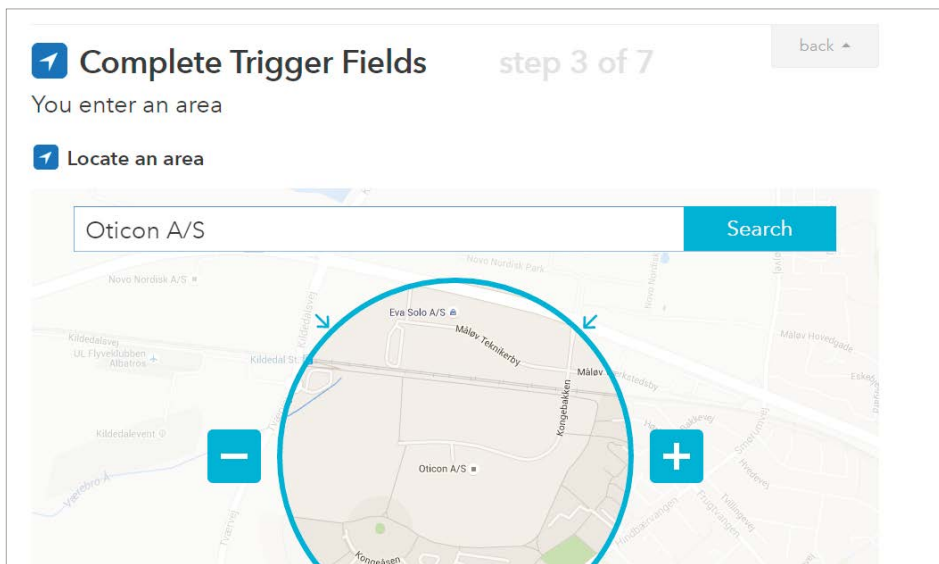


6

Press the big blue 'that'




7

Type 'Oticon' in the search field and choose 'Oticon ON' by clicking on it

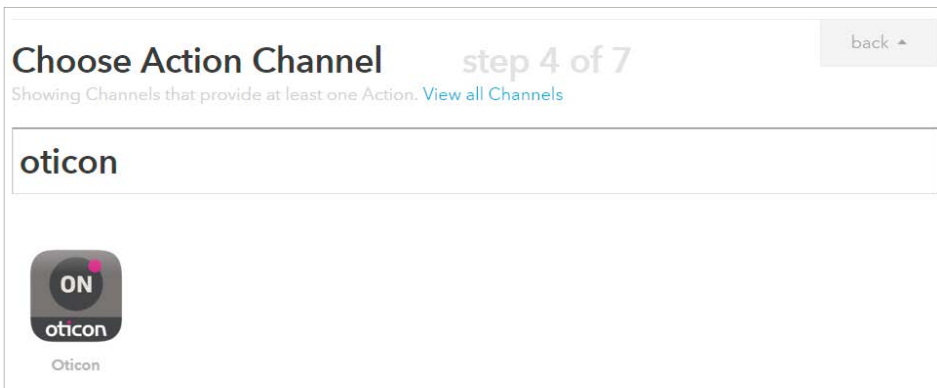


8

Click on Connect. In the pop-up window: Log in with your Oticon ON account.

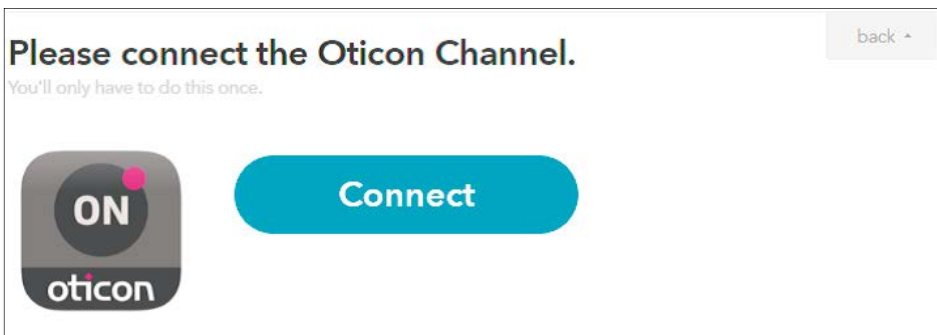

9

Choose the action 'Set a program' by clicking on it

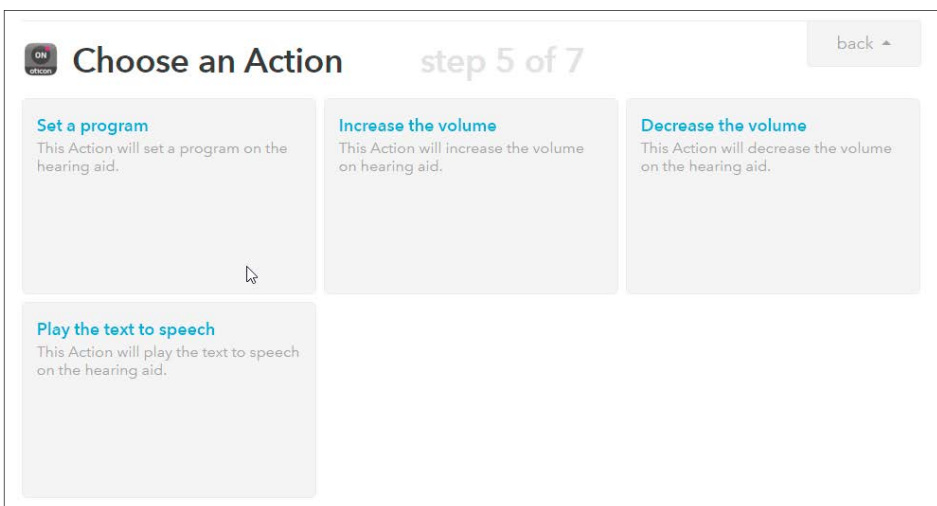

10

Select the wanted program from the list. When finished, click on 'Create Action'

Complete Action Fields step 6 of 7 back

Set a program

Program ID to be set

Please select

Program ID to be set on the hearing aid (e.g. RESTAURANT)

Please select

- P1
- P2
- P3
- TV1
- iPhone Mic

Create Action

11

Finally click on 'Create Recipe' to create and activate the recipe which switches to another program when you enter a specific location.

Recipe Title

If You enter an area, then set a program

use '#' to add tags

Receive notifications when this Recipe runs

Create Recipe