

Oticon Opn Play™ classroom connectivity

A full range of easy-to-use solutions for the classroom

ConnectClip remote microphone

ConnectClip connects to integrated 2.4 GHz receiver



Universal ear-level options

- Connect to Oticon Amigo FM
- Connect to transmitters from other manufacturers



FM10 adapter



Amigo R2 receiver



Universal neckloop receiver



Amigo Arc

Oticon Amigo FM

Dedicated FM receiver



Amigo R12G2

Oticon Opn Play™ classroom connectivity



Oticon Amigo FM - ear-level

Setting the system up requires FM transmitter and an FM receiver. Connect FM receiver to the hearing aid directly using the FM adapter battery drawer and switch on the FM transmitter to activate the FM system.

- To attach either Oticon Amigo R12G2 or the FM 10 adapter to Oticon Opn Play BTE PP, the standard battery drawer needs to be replaced with the FM adapter battery drawer.
- The FM adapter battery drawer and Oticon Amigo R12G2 are available to match instrument colors.
- The FM adapter battery drawer can be made tamper resistant by inserting the special tamper-resistant left/right color markers.



FM adapter battery drawer



Amigo T30/T31 FM transmitter



Amigo R12G2 Dedicated FM receiver



Amigo R2 universal FM receiver



FM 10 Universal FM adapter

Let children connect easily to the world around them

Connectivity options

Stereo streaming (2.4 GHz)	•	•	•	•
Oticon ON App	•	•	•	•
ConnectClip	•	•	•	•
Remote Control 3.0	•	•	•	•
TV Adapter 3.0	•	•	•	•
Phone Adapter 2.0	•	•	•	•

A full range of easy-to-use solutions for the classroom

Classroom receiver options

Integrated 2.4 GHz receiver	•	•	•	•
Dedicated FM receiver	Amigo R12G2	-	-	-
Universal ear-level FM receiver	Amigo R2	-	-	-
Universal neckloop receiver	Amigo Arc	Amigo Arc	Amigo Arc	-

Integrated 2.4 GHz solution

Completely integrated solution with the built-in 2.4 GHz Bluetooth™ low energy receiver in every Oticon Opn Play hearing aid, streams speech seamlessly from the remote microphone in **ConnectClip**.

Dedicated FM solution

The dedicated **Oticon Amigo R12G2** FM receiver integrates smoothly with the design of the Oticon Opn Play BTE PP. This design integrated solution allows connection to Oticon Amigo FM transmitters.

Universal ear-level solution

Oticon Opn Play BTE PP ensures full compatibility with existing classroom solutions through the **FM 10 adapter**. The student can use a universal ear-level solution by attaching any universal receiver to the FM 10 adapter.

Universal neckloop solution

Oticon Opn Play models with telecoil ensure compatibility with Oticon Amigo FM transmitters through the **Oticon Amigo Arc** FM neckloop receiver. This universal neckloop solution offers a common FM solution, and only requires one FM receiver per student.