



	More 1	More 2	More 3	
<b>Speech Understanding</b>	MoreSound Intelligence™	Level 1	Level 2	Level 3
	- Environment configuration	5 Options	5 Options	3 Options
	- Virtual Outer Ear	3 Configurations	1 Configuration	1 Configuration
	- Spatial Balancer	100%	60%	60%
	- Neural Noise Suppression, Difficult / Easy	10 dB / 4 dB	6 dB / 2 dB	6 dB / 0 dB
	- Sound Enhancer	3 Configurations	2 Configurations	1 Configuration
	MoreSound Amplifier™	•	•	•
	Feedback Prevention	MoreSound Optimizer™ & Feedback shield	MoreSound Optimizer™ & Feedback shield	MoreSound Optimizer™ & Feedback shield
	Spatial Sound™	4 Estimators	2 Estimators	2 Estimators
	Soft Speech Booster	•	•	•
Frequency lowering	Speech Rescue™	Speech Rescue™	Speech Rescue™	
<b>Sound Quality</b>	Clear Dynamics	•	•	-
	Better-Ear Priority	•	•	-
	Fitting Bandwidth*	10 kHz	8 kHz	8 kHz
	Bass Boost (streaming)	•	•	•
	Processing Channels	64	48	48
<b>Listening Comfort</b>	Transient Noise Management	4 configurations	3 configurations	3 configurations
	Wind Noise Management	•	•	•
<b>Personalization &amp; Optimizing Fitting</b>	Fitting Bands	24	20	18
	Multiple Directionality options	•	•	•
	Adaptation Management	•	•	•
	Fitting Formulas	VAC+, NAL-NL1/ NAL-NL2, DSL 5.0	VAC+, NAL-NL1/ NAL-NL2, DSL 5.0	VAC+, NAL-NL1/ NAL-NL2, DSL 5.0
<b>Connecting to the world</b>	Hands-free communication**	•	•	•
	Direct streaming***	•	•	•
	Oticon ON app & Oticon RemoteCare app	•	•	•
	ConnectClip	•	•	•
	EduMic	•	•	•
	Remote Control 3.0	•	•	•
	TV Adapter 3.0	•	•	•
	Phone Adapter 2.0	•	•	•
	Tinnitus SoundSupport™	•	•	•
	CROS/BiCROS support	•	•	•

\* Bandwidth accessible for gain adjustments during fitting  
 \*\* Available for Oticon More from FW 1.3 with select iPhone models  
 \*\*\* From iPhone®, iPad®, iPod touch®, and select Android™ devices

**Operating and charging conditions**  
 Temperature: +5°C to +40°C (41°F to 104°F)  
 Relative humidity: 5% to 93%, non-condensing  
 Atmospheric pressure: 700 hPa to 1060 hPa

**Storage and transportation conditions**  
 Temperature and humidity should not exceed the below limits for extended periods during transportation and storage.

**Transport**  
 Temperature: -20°C to +60°C (-4°F to 140°F)  
 Relative humidity: 5% to 93%, non-condensing  
 Atmospheric pressure: 700 hPa to 1060 hPa

**Storage**  
 Temperature: -20°C to +30°C (-4°F to 86°F)  
 Relative humidity: 5% to 93%, non-condensing  
 Atmospheric pressure: 700 hPa to 1060 hPa

Oticon More™ miniBTE R is small in size and fits most ears. It is powered by a rechargeable lithium-ion battery. The style features telecoil, and a single push-button. It is a Made for iPhone® hearing aid and compatible with the new Android protocol for Audio Streaming for Hearing Aids (ASHA) - making it possible to stream directly from iPhone, iPad®, iPod touch® and select Android™ devices.

MoreSound Intelligence™ creates a more precise and natural representation of individual sounds with clearer and more distinct contrasts providing access to all relevant sounds.

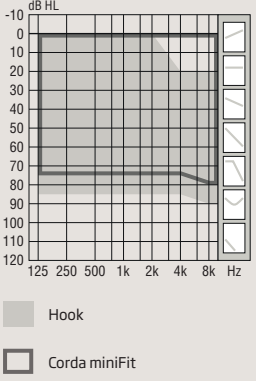
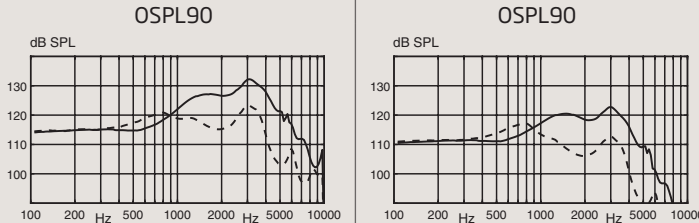
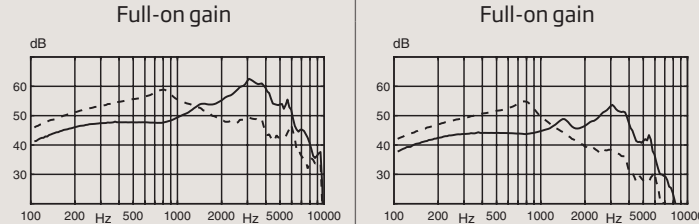

MoreSound Amplifier™ analyzes details in sound, and optimally amplifies them for the brain to have access to relevant information.

Oticon More is built on the innovative Polaris™ platform, which uses a Deep Neural Network to rapidly and optimally manage incoming sounds based on individual needs.

Apple, the Apple logo, iPhone, iPad, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.



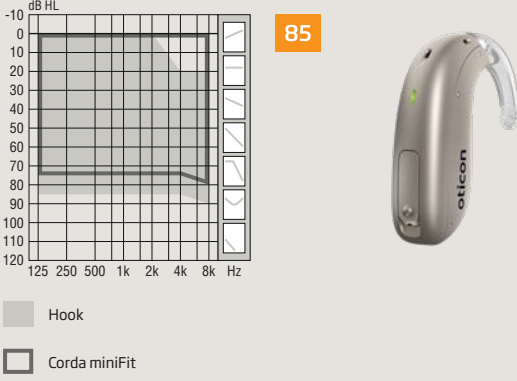
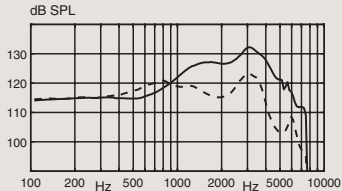
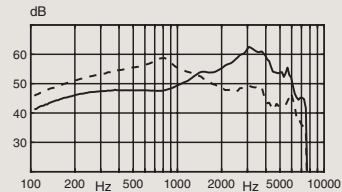
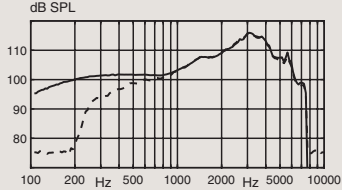
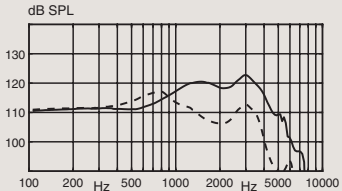
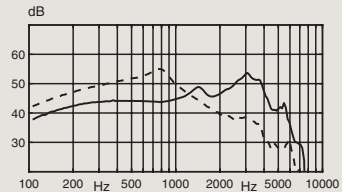
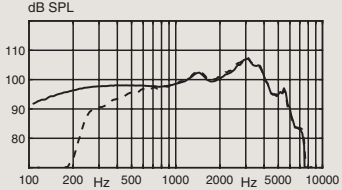
For information on compatibility, please visit [www.oticon.com/support/compatibility](http://www.oticon.com/support/compatibility)

		<b>Ear Simulator</b> Measured according to IEC 60118-0:1983/AMD1:1994, IEC 60118-0:2015, IEC 60118-1:1995+AMD1:1998 CSV and IEC 60318-4:2010	<b>2CC Coupler</b> Measured according to ANSI S3.22-2014, IEC 60118-0:2015 and IEC 60318-5:2006
 <p><b>Technical information</b> Omnidirectional mode is used unless otherwise stated.</p>			
			
			
OSPL90	Peak	132 (123 <sup>1</sup> ) dB SPL	123 (117 <sup>1</sup> ) dB SPL
	1600 Hz	127 (116 <sup>1</sup> ) dB SPL	120 (108 <sup>1</sup> ) dB SPL
	HFA-OSPL90	126 (118 <sup>1</sup> ) dB SPL	119 (110 <sup>1</sup> ) dB SPL
Full-on gain <sup>2</sup>	Peak	63 (59 <sup>1</sup> ) dB	54 (55 <sup>1</sup> ) dB
	1600 Hz	54 (51 <sup>1</sup> ) dB	47 (43 <sup>1</sup> ) dB
	HFA-FOG	54 (51 <sup>1</sup> ) dB	47 (43 <sup>1</sup> ) dB
Reference test gain		47 dB	41 dB
Frequency range		100-9500 Hz	100-7300 Hz
Telecoil output (1600 Hz)	1 mA/m field	85 dB SPL	-
	10 mA/m field	105 dB SPL	-
	SPLITS L/R	-	99/99 dB SPL
Total harmonic distortion (Input 70 dB SPL)	500 Hz	< 4 %	< 4 %
	800 Hz	< 4 %	< 3 %
	1600 Hz	< 2 %	< 2 %
Equivalent input noise level	Omni	19 dB SPL	17 dB SPL
	Dir	30 dB SPL	31 dB SPL
Battery		Lithium-ion	Lithium-ion
Expected operating time, hours <sup>3</sup>		24	

1) For instruments fitted with Corda miniFit

2) Measured with the gain control of the hearing aid set to its full-on position minus 20 dB and with an input SPL of 70 dB. This is to obtain a gain response equal to the full-on gain response from e.g. IEC 60118-0+A1:1994 but without influence of feedback.

3) Expected operating time for rechargeable battery depends on use pattern, active feature set, hearing loss, sound environment, battery age and use of wireless accessories.

		<b>Ear Simulator</b> Measured according to IEC 60118-0:1983/AMD1:1994, IEC 60118-0:2015, IEC 60118-1:1995+AMD1:1998 CSV and IEC 60318-4:2010	<b>2CC Coupler</b> Measured according to ANSI S3.22-2014, IEC 60118-0:2015 and IEC 60318-5:2006
 <p><b>85</b></p> <p>Legend:                      ■ Hook                      □ Corda miniFit</p> <p><b>Technical information</b>                      Omnidirectional mode is used unless otherwise stated.</p>		<p><b>OSPL90</b></p>  <p><b>Full-on gain</b></p>  <p><b>Frequency response</b></p>  <p>Legend:                      — Standard tube                      - - - Thin tube (size 0.9)</p> <p>Legend:                      — Acoustic input: 60 dB SPL                      - - - Magnetic input: 31.6 mA/m</p>	<p><b>OSPL90</b></p>  <p><b>Full-on gain</b></p>  <p><b>Frequency response</b></p>  <p>Legend:                      — Standard tube                      - - - Thin tube (size 0.9)</p> <p>Legend:                      — Acoustic input: 60 dB SPL                      - - - Magnetic input: 31.6 mA/m</p>
OSPL90	Peak	132 (123 <sup>1</sup> ) dB SPL	123 (117 <sup>1</sup> ) dB SPL
	1600 Hz	127 (116 <sup>1</sup> ) dB SPL	120 (108 <sup>1</sup> ) dB SPL
	HFA-OSPL90	126 (118 <sup>1</sup> ) dB SPL	119 (110 <sup>1</sup> ) dB SPL
Full-on gain <sup>2</sup>	Peak	63 (59 <sup>1</sup> ) dB	54 (55 <sup>1</sup> ) dB
	1600 Hz	54 (51 <sup>1</sup> ) dB	47 (43 <sup>1</sup> ) dB
	HFA-FOG	54 (51 <sup>1</sup> ) dB	47 (43 <sup>1</sup> ) dB
Reference test gain		47 dB	41 dB
Frequency range		100-7500 Hz	100-7300 Hz
Telecoil output (1600 Hz)	1 mA/m field	85 dB SPL	-
	10 mA/m field	105 dB SPL	-
	SPLITS L/R	-	99/99 dB SPL
Total harmonic distortion (Input 70 dB SPL)	500 Hz	< 4 %	< 4 %
	800 Hz	< 4 %	< 3 %
	1600 Hz	< 2 %	< 2 %
Equivalent input noise level	Omni	19 dB SPL	17 dB SPL
	Dir	30 dB SPL	32 dB SPL
Battery		Lithium-ion	Lithium-ion
Expected operating time, hours <sup>3</sup>		24	

1) For instruments fitted with Corda miniFit

2) Measured with the gain control of the hearing aid set to its full-on position minus 20 dB and with an input SPL of 70 dB. This is to obtain a gain response equal to the full-on gain response from e.g. IEC 60118-0+A1:1994 but without influence of feedback.

3) Expected operating time for rechargeable battery depends on use pattern, active feature set, hearing loss, sound environment, battery age and use of wireless accessories.

**Headquarters**  
Oticon A/S  
Kongebakken 9  
DK-2765 Smørum  
Denmark



SBO Hearing A/S  
Kongebakken 9  
DK-2765 Smørum  
Denmark