

# Opn Firmware 4 **Update**



**With the introduction of Firmware 4 for Oticon Opn hearing aids (June 2017), new features such as Tinnitus SoundSupport™ and Speech Rescue™ LX are added.** In addition, improvements for wireless programming and instrument streaming are included with Firmware 4. These updates allow even more users to experience the benefits of Oticon Opn: listening with less effort, better recall, and better speech understanding.

**Please note that with the Firmware 4 update for Oticon Opn, the following streaming scenarios may still occur, pending further iOS updates:**

- Audio streaming and phone calls “bounce” between ears;
- Momentary disconnect/reconnect with streamed audio/phone calls;
- Intermittent ticking may be heard with Fitbit or Apple Watch Connections;
- When using a map application (like Google Maps), the first part of the audio directions may be missed;
- When un-pairing hearing aids from iPhone, you must ensure that Bluetooth is enabled.

**As a reminder, after a hearing aid’s Firmware is updated:**

- Clear pairings on Remote Control 3.0 and in the Oticon ON App, then re-pair with the hearing aids after the firmware update has been completed.

Oticon engineers work closely with Apple engineers to constantly improve hearing aid streaming using the iOS platform. **Solutions to these scenarios above are expected with the release of iOS 11.**

Apple, the Apple logo, iPhone, iPad, iPod touch, and Apple Watch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.