Instructions for Use

INSTRUCTIONS FOR USE - GB
GEBRAUCHSANWEISUNG - DE
INSTRUCTIONS D'UTILISATION - FR
BRUKSANVISNING - SE
ISTRUZIONI PER L'USO - IT
BRUGSANVISNING - DK
MANUAL DE INSTRUCCIONES - ES
INSTRUKCJA UŻYCIA - PL

Stereo Amplifier AP70

8100457 D-0101839-C – 2020/03

Instructions for Use - GB

Stereo Amplifier AP70

Table of Contents

1.	INTRODUCTION	1
1.1	ABOUT THIS MANUAL	1
2.	UNPACKING/INSPECTION	3
2.1	UNPACKING AND INSPECTION.....	3
2.2	REPORTING IMPERFECTIONS	3
2.3	SAFETY REGULATIONS.....	3
2.4	LIST OF SYMBOLS.....	5
3.	OPERATION INSTRUCTION	7
3.1	CONNECTION PANEL DIRECTORY	7
3.2	AP70 - CONNECTION PANEL DICTIONARY	7
3.3	AP70 - OPERATION PANEL DICTIONARY	7
3.4	SETUP INSTRUCTION.....	8
4.	MAINTENANCE	11
4.1	GENERAL MAINTENANCE PROCEDURES.....	11
4.2	HOW TO CLEAN RADIOEAR PRODUCTS	11
4.3	CONCERNING REPAIR	11
4.4	WARRANTY	12
5.	TECHNICAL SPECIFICATIONS	13
5.1	PIN ASSIGNMENT	14

1. Introduction

1.1 About this Manual

This manual is valid for the AP70. The product is manufactured by:

Radioear A/S
Audiometer Allé 1
5500 Middelfart
Denmark
Tel.: +45 6371 3555
Fax: +45 6371 3522
E-mail: info@Radioear.com
Web: www.Radioear.com

AP70 Instructions for Use - English

Date: 2020/03

Page 2/14

2. Unpacking/Inspection

2.1 Unpacking and inspection

Check Box and Contents for Damage

When the unit is received please check the shipping box for any rough handling or damage. If the box is damaged it should be kept until the contents of the shipment have been checked mechanically and electrically. If the device is faulty, please contact the nearest service office. Keep the shipping material for the carrier's inspection and insurance claim.

Store Carton for future Shipment

The unit comes in its own shipping carton which is specially designed for it. Please store this carton; it will be needed if the device has to be returned for service. If service is required please contact your nearest sales and service office.

Contents of Shipment

As standard, this unit is delivered with the following:

Included parts	Mains Cable ID numbers! Input cables Speaker cables Remote cable
----------------	---

Reporting Imperfections:

Inspect before connection:

Prior to connecting the device to the Power Supply it should be inspected once more for damage. All of the cabinet and the accessories should be checked visually for scratches and missing parts.

Report any faults immediately:

Any missing part or malfunction should be reported immediately to the supplier of the unit together with the invoice, serial number and detailed report of the problem. In the back of this manual a "Return Report" can be found, where you can describe the problem.

Please use "Return Report":

Please realize that if the service engineer does not know what the problem is to look for, he may not find it. Therefore using the Return Record will be of great help to us and at the same time it is guaranteed that the correction to your problem will be to satisfaction.

2.2 Important Safety Instructions

Read these instructions

This device is specified to comply with the international standard IEC 60065

To avoid the risk of electric shock, this equipment must only be connected to a supply main with protective earth."

- 1) Read these instructions.
- 2) Keep these instructions.
- 3) Heed all warnings.
- 4) Follow all instructions.
- 5) Do not use this apparatus near water.
- 6) Clean only with dry cloth.
- 7) Do not block any ventilation openings. Install in accordance with the manufacturer's instructions.
- 8) Do not install near any heat sources such as radiators, heat registers, stoves, or other apparatus (including amplifiers) that produce heat.

AP70 Instructions for Use - English

Date: 2020/03

Page 4/14

- 9) Do not defeat the safety purpose of the polarized or grounding-type plug. A polarized plug has two blades with one wider than the other. A grounding type plug has two blades and a third grounding prong. The wide blade or the third prong are provided for your safety. If the provided plug does not fit into your outlet, consult an electrician for replacement of the obsolete outlet.
- 10) Protect the power cord from being walked on or pinched particularly at plugs, convenience receptacles, and the point where they exit from the apparatus.
- 11) Only use attachments/accessories specified by the manufacturer.
- 12) Use only with the cart, stand, tripod, bracket, or table specified by the manufacturer, or sold with the apparatus. When a cart is used, use caution when moving the cart/apparatus combination to avoid injury from tip-over.
- 13) Unplug this apparatus during lightning storms or when unused for long periods of time.
- 14) Refer all servicing to qualified service personnel. Servicing is required when the apparatus has been damaged in any way, such as power-supply cord or plug is damaged, liquid has been spilled or objects have fallen into the apparatus, the apparatus has been exposed to rain or moisture, does not operate normally, or has been dropped

NOTE: The mains plug is intended to disconnect the device from mains

NOTE. This amplifier is designed to fulfill the requirements of the medical standard IEC/ES 60601-1 when connected to medical equipment. That means it can be connected to an audiometer without further electrical protection or isolation.

No modification of this equipment is allowed without Radioear authorization.

Radioear will make available on request circuit diagrams, component part lists, descriptions, calibration instructions, or other information that will assist service personnel to repair those parts of this equipment that are designated by the Radioear as repairable by service personnel.

The apparatus shall not be exposed to dripping or splashing and that no objects filled with liquids, such as vases, shall be placed on the apparatus.

AP70 Instructions for Use - English

Date: 2020/03

Page 5/14

2.3 List of Symbols

Symbol	Explanation
	Refer to instruction manual
	<p>WEEE Declaration This symbol indicates that when the end-user wishes to discard this product, it must be sent to separate collection facilities for recovery and recycling.</p> <p>Within the European Union it is illegal to dispose electric and electronic waste as unsorted municipal waste. Electric and electronic waste may contain hazardous substances and therefore has to be collected separately. Such products will be marked with the crossed-out wheeled bin shown below. The cooperation of the user is important in order to ensure a high level of reuse and recycling of electric and electronic waste. Failing to recycle such waste products in an appropriate way may endanger the environment and consequently the health of human being</p> <p>This device complies to: RoHS Directive (2011/65/EU) WEEE Directive (2002/96/EC)</p>
	<p>The CE Marking indicates that the product complies with the essential requirements of the relevant European health, safety and environmental protection legislation.</p> <p>This device conforms to: EMC Directive (2004/108/EC) Low Voltage Directive (2006/95/EC)</p>
	Year of manufacture
 CAUTION RISK OF ELECTRIC SHOCK DO NOT OPEN	<p>Presence of uninsulated dangerous voltage inside the device.</p> <p>Never open the device as there is a risk of electrical shock. There are no user serviceable parts inside. Refer any service to authorized RadioEar service partner.</p> <p>Read and follow the safety and operating instructions</p>

AP70 Instructions for Use - English

Date: 2020/03

Page 6/14


AP70 Instructions for Use - English

Date: 2020/03

Page 7/14

3. Operation Instruction

3.1 Connection Panel Directory

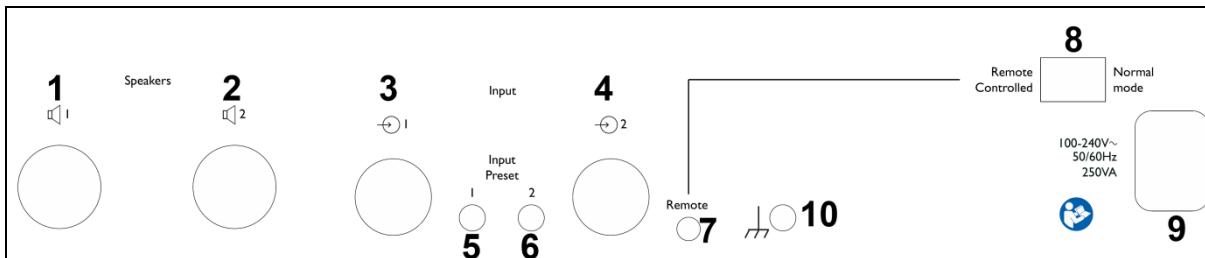3.2 AP70 - Connection Panel Dictionary

Position:	Symbol:	Function:
1	Out	Plug for speaker CH1
2	Out	Plug for speaker CH2
3	In	Plug for input signal CH1
4	In	Plug for input signal CH 2
5	Pre-set	Input sensitivity adjustment for CH1
6	Pre-set	Input sensitivity adjustment for CH 2
7	Remote	Input
8	Remote	Remote power on Controlled / Normal mode
9	Power	Plug for Remote power on
10	+	Ground terminal for setup calibration

Input Level

The input sensitivity for CH1 and CH2 are controlled by the buttons (5) and (6) "Preset" on the connection panel. Do not adjust the buttons (5) and (6) when the system is calibrated.

Remote power on

The amplifier can be switched on and off remotely. In the position "Normal mode" (8) can only be switched on and off by the Main switch (10). In the position "Remote controlled" the amplifier can be turned on by applying 5VDC at the Remote plug (9), 0VDC will turn off the amplifier.

3.3 AP70 - Operation Panel Dictionary

AP70 Power Amplifier

RADIOEAR

Position:	Symbol:	Function:
10	Power	Mains power switch
11	⚠	Overload / Over temperature indicator
12	Power	Power indicator

AP70 Instructions for Use - English

Date: 2020/03


Page 8/14

Indicator guide


Indicator	Information	Action
Power	Normal mode	
	Power	Speaker protection Lower the input level
	Over heat	Turn off the amplifier, let it cool down and turn it on

3.4 Setup instruction

1. Connect the amplifier to the audiometer with the RCA-XLR cable
2. Connect the audiometer's and the amplifier's power plugs to a power socket with ground connection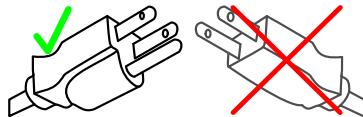

3. Adjust the audiometer to maximum unclipping output
 4. Connect the one speaker plug into the amplifier (without speaker)
 5. Connect an oscilloscope in AC-mode at the one pole at CH 2 speaker cable output.
- The measurement of the output must be made between the GND-point and the one pole at the speaker connector, as shown below.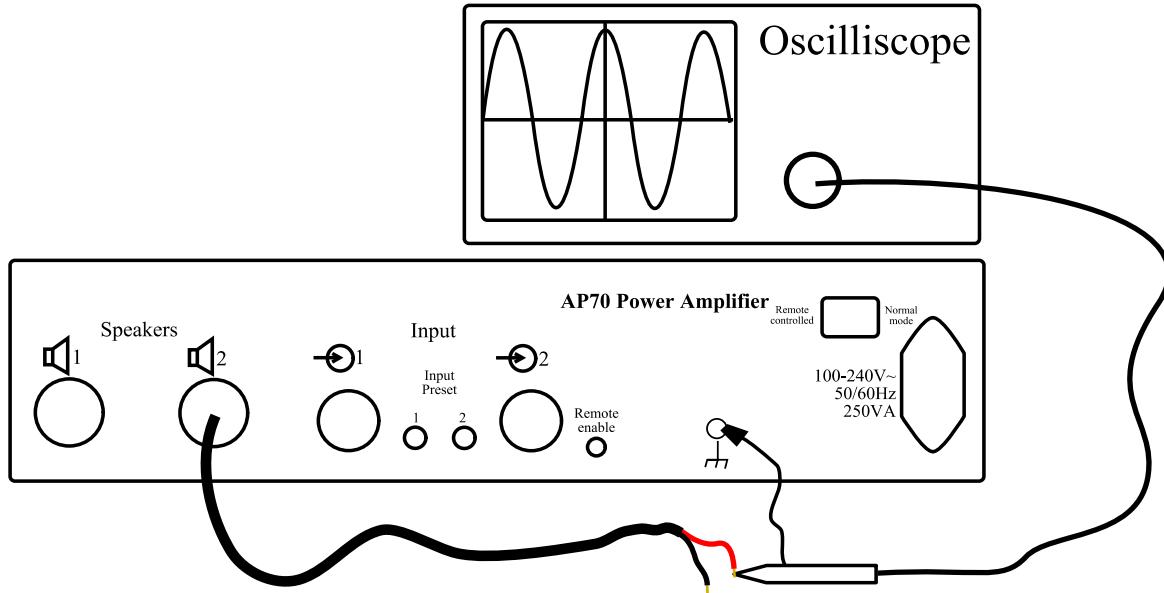


Do not connect the GND alligator test lead of the oscilloscope to the output of the amplifier, only to the GND terminal at the amplifier.


AP70 Instructions for Use - English

Date: 2020/03

Page 9/14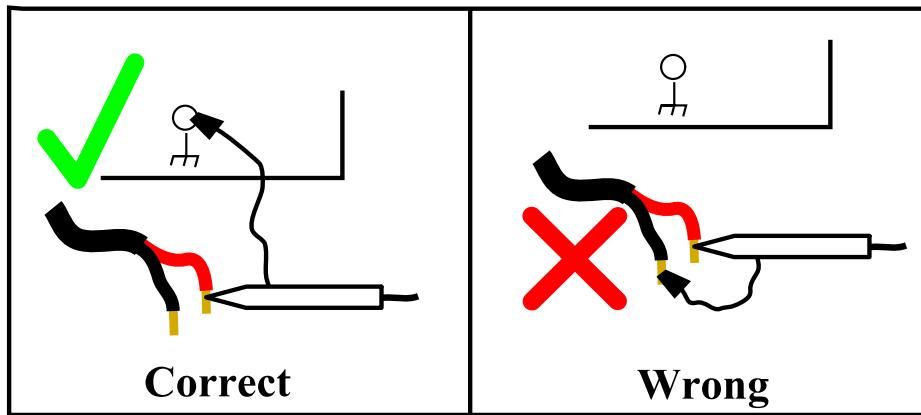

6. Adjust the Input preset 2 clockwise to just below the clipping limit.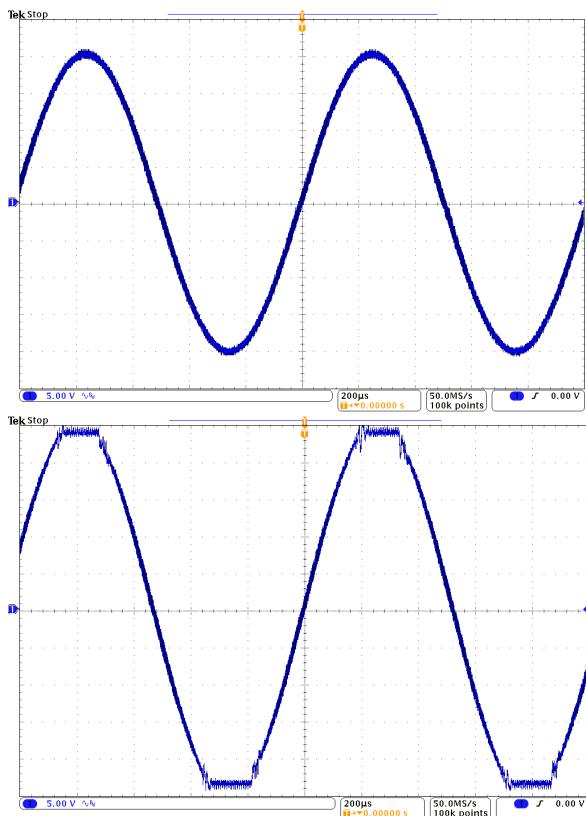

7. Repeat point 4. & 5. at CH1
8. Turn off the audiometer and amplifier
9. Connect the speakers and calibrate the system according to the audiometer's instruction

AP70 Instructions for Use - English

Date: 2020/03

Page 10/14

4. Maintenance

4.1 General Maintenance Procedures

The performance and safety of the instrument will be kept if the following recommendations for care and maintenance are observed:

- The instrument must go through at least one annual overhaul, to ensure that the acoustical, electrical and mechanical properties are correct. This should be made by an authorised workshop in order to guarantee proper service and repair as Radioear provides the necessary circuit diagrams etc. to these repair shops.
- To ensure that the reliability of the instrument is kept, it is recommended that the operator at short intervals, for instance once a day, perform a test on a person with known data. This person could be the operator him/herself.
- After each examination of a patient, it should be ensured that there is no contamination on the parts in connection with the patient. General precautions must be observed in order to avoid that disease from one patient is conducted to others. If ear cushions or ear tips are contaminated, it is strongly recommended to remove them from the transducer before they are cleaned. By frequent cleaning water should be used, but by severe contamination it may be necessary to use a disinfectant. The use of organic solvents and aromatic oils must be avoided.

4.2 How to clean Radioear products

CAUTION

- Before cleaning always switch off and disconnect from the power supply
- Use a soft cloth lightly dampened with cleaning solution to clean all exposed surfaces
- Do not allow liquid to come in contact with the metal parts inside the earphones / headphones
- Do not autoclave, sterilize or immerse the instrument or accessory in any fluid
- Do not use hard or pointed objects to clean any part of the instrument or accessory
- Do not let parts that have been in contact with fluids dry before cleaning
- Rubber ear-tips or foam ear-tips are single use components
- Ensure that isopropyl alcohol does not come into contact with any screens on the instruments
- Ensure that isopropyl alcohol does not come into contact with any silicone tubes or rubber parts

Recommended cleaning and disinfection solutions:

- Warm water with mild, nonabrasive cleaning solution (soap)
- Normal hospital bactericides
- 70% isopropyl alcohol only on hard cover surfaces

Procedure

- Clean the instrument by wiping outer case with a lint free cloth lightly dampened in cleaning solution.
- Clean cushions and patient hand switch and other parts with a lint free cloth lightly dampened in cleaning solution.
- Make sure not to get moisture in the speaker portion of the earphones and similar parts

4.3 Concerning Repair

Radioear is only considered to be responsible for the validity of the CE marking, effects on safety, reliability and performance of the equipment if:

1. assembly operations, extensions, readjustments, modifications or repairs are carried out by authorised persons,
2. a 1 year service interval is maintained
3. the electrical installation of the relevant room complies with the appropriate requirements, and
4. the equipment is used by authorised personnel in accordance with the documentation supplied by Radioear.

AP70 Instructions for Use - English

Date: 2020/03

Page 12/14

It is important that the customer (agent) fills out the RETURN REPORT every time a problem arises and sends it to Radioear. This should also be done every time an instrument is returned to Radioear. (This of course also applies in the unthinkable worst case of death or serious deterioration to patient or user).

4.4 Warranty

RADIOEAR warrants that:

- This product is free from defects in material and workmanship under normal use and service for a period of 24 months from the date of delivery by Radioear to the first purchaser.
- Accessories are free from defects in material and workmanship under normal use and service for a period of ninety (90) days from the date of delivery by Radioear to the first purchaser.

If any product requires service during the applicable warranty period, the purchaser should communicate directly with the local Radioear service centre to determine the appropriate repair facility. Repair or replacement will be carried out at Radioear' expense, subject to the terms of this warranty. The product requiring service should be returned promptly, properly packed, and postage prepaid. Loss or damage in return shipment to Radioear shall be at purchaser's risk.

In no event shall Radioear be liable for any incidental, indirect or consequential damages in connection with the purchase or use of any Radioear product.

This shall apply solely to the original purchaser. This warranty shall not apply to any subsequent owner or holder of the product. Furthermore, this warranty shall not apply to, and Radioear shall not be responsible for, any loss arising in connection with the purchase or use of any Radioear product that has been:

- repaired by anyone other than an authorized Radioear service representative;
- altered in any way so as, in Radioear judgement, to affect its stability or reliability;
- subject to misuse or negligence or accident, or which has had the serial or lot number altered, effaced or removed; or
- improperly maintained or used in any manner other than in accordance with the instructions furnished by Radioear.

This warranty is in lieu of all other warranties, express or implied, and of all other obligations or liabilities of Radioear, and Radioear does not give or grant, directly or indirectly, the authority to any representative or other person to assume on behalf of Radioear any other liability in connection with the sale of Radioear products.

RADIOEAR DISCLAIMS ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FOR FUNCTION OF FITNESS FOR A PARTICULAR PURPOSE OR APPLICATION

5. Technical Specifications

Standards:

Safety IEC60065:2002, UL60065:2007, CAN/CSA C22.2 No.60065:03, CLASS I
EMC CISPR11, EN/IEC61000-4-2

Mains nominal input voltage:

100 - 240V~

Mains frequency:

50 - 60Hz

Consumption (Nominal):

200 Watt

Fuse:

1 x 6,3AT Located on the main PCB

Output Overload Protection:

Short circuit protected.

Output Power, one channel alone:

110WRMS continues for 10 minutes at 8Ω.
140WRMS continues for 10 minutes at 4Ω.

Output Power, both channels active:

2x70WRMS continues for 10 minutes at 8Ω.
2x70 WRMS continues for 10 minutes at 4Ω. (2x100W in 4Ω <10s)

Input:

Balanced and isolated

Input impedance: 2kΩ

Max sensitivity: 0.3VRMS for max output, with adjustable with pre-set potentiometers.

Max input voltage: 7VRMS

Frequency Range:

20-20.000 Hz ± 3dB with 8Ω load.

Noise:

>98dB signal/noise ratio, A-weighted measured.

Channel Separation:

Typical > 60dB undistorted output on opposite channel.

Total Harmonic Distortion:

< 0.2 % @ 70W..

Dimensions (W x D x H):

40 x 21.5 x 8.2 cm / 15.8 x 8.5 x 3.2 inches

Weight:

3.9kg / 8.6lb

Environment:

Operation Temperature: 15 - 35°C

Operation Humidity: 30 - 90% RH, Non-condensing

Transportation and storage of this unit should be within the following environmental conditions:

Transportation: -20° to 50°C

Storage: 0° to 50°C




Humidity: 10 - 95% RH, Non-condensing. Keep dry.

AP70 Instructions for Use - English

Date: 2020/03

Page 14/14

5.1 Pin Assignment

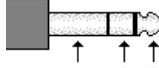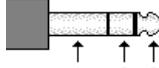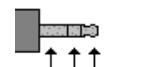Socket	Connector	Pin 1	Pin 2	Pin 3
Mains	 IEC C13	Live	Neutral	Earth
Speakers 1		Signal (1+)	Signal (1-)	-
Speakers 2	 SpeakON 2 pol			
Input 1	 or  XLR / 6.3mm Jack		Signal (+)	Signal (-)
Input 2	 XLR / 6.3mm Jack	Signal (-)	-	Signal (+)
Remote	 3.5mm Jack	Ground	-	Remote signal (5 VDC)

Input & Remote: Mono versions of the jack plugs can be used without problems.

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