RADIOEAR DD45 HEADSET

Instructions for Use – UK/US
The DD45 is a supra-aural headset that can be used for audiometry testing for patients in all ages. The device converts electrical signals from the microphone into air transmitted acoustic signals and couple these signals to the ear of the patient. The device is intended to be used by an audiologist, hearing healthcare professional, or trained technician.

Warning: Only connect to IEC 60601-1 and IEC 60601-1-2 approved devices. Do not use a defective device.

Caution: Before use, make sure the headset is calibrated to the audiometric equipment it is to be used with.

Cleaning instruction

To clean the headphones, use a soft, damp cloth. The use of organic solvents and aromatic oils must be avoided. If necessary, clean the cushions with a lint free cloth lightly dampened in cleaning solution. Make sure not to get moisture in the speaker.

Specifications	
• Standards:	IEC 60645-1:2017, ANSI S3.6-2018, ISO 389-1:2017 & ISO 389-4:1994
• Earphone receiver:	Dynamic moving coil, custom built for audiometry
• Nominal impedance:	10 Ω
• Frequency response:	100 Hz – 8,000 Hz
• Sensitivity:	106,7 dB SPL bei 1 kHz, 1 mW
• Distortion:	< 1% @ 100 Hz, 1.000 Hz & 6.000 Hz
• Max. continuous power:	1 W
• Headband tension:	4,5 N ± 0,5 N
• Weight:	approx. 380g
• Cable length:	0,38 m to 5 m, depending on variant.
• variants of DD45:	DD45C (Contra) and DD45S (Shielded)

Compliant to RoHS 2011/65/EU incl. 2015/863

See www.radioear.us for range of spare parts and accessories

In case of serious incident in relation to the use of the device, the incident must immediately be reported to DGS Diagnostics A/S and the local national competent authority.

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