RADIOEAR IP30 INSERT PHONE HEADSET

Instructions for Use - UK/US

The IP30 Insert phone is designed for contralateral procedures within audiometry for patients in all ages. The device converts electrical signals from the audiometer into air transmitted acoustic signals and couple these signals to the ear of the patient. The device is intended to be used by an audiologist, hearing healthcare professional, or trained technician.

Warning: Only connect to IEC 60601-1 and IEC 60601-1-2 approved devices. Do not use a defective device.

Caution: Before use, make sure the headset is calibrated to the audiomeric equipment it is to be used with.

Cleaning instruction

To clean the insert earphone, use a soft, damp cloth. The use of organic solvents and aromatic oils must be avoided. If necessary, clean the device with a lint free cloth lightly dampened in cleaning solution. Make sure not to get moisture in the speaker.

Specifications

• Standards:	IEC 60645-1:2017, ANSI S3.6-2010 & ISO 389-2:1994
• Nominal impedance:	10 Ω or 50 Ω
• Frequency response:	125 Hz – 8,000 Hz
• Sensitivity:	Typical 103 dB in HA-2 coupler (IEC 60318-5:2006) Typical 109 dB in ear simulator (IEC 60318-4:2010) @ 0,1 VRMS (10 Ω) @ 0,2 VRMS (50 Ω)
• Weight:	18 grams (0,634 oz)
• Cable length:	0,4 m to 5 m, depending on variant

Compliant to RoHS 2011/65/EU incl. 2015/863

See www.radioear.us for range of spare parts and accessories

In case of serious incident in relation to the use of the device, the incident must immediately be reported to DGS Diagnostics A/S and the local national competent authority.

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