



Remote Fitting with your hearing care professional

Please check the following guidelines that will help you get the most out of your Remote Fitting appointment. For more information, please refer to the Remote Fitting instructions for use accessible on the Sonic SoundLink Connect app or seek help from your hearing care professional.

Before your Remote Fitting appointment:

- ❑ Ensure Bluetooth® is activated on your smartphone or tablet.
- ❑ Pair your hearing aids to your smartphone or tablet if not already done.
- ❑ Ensure your smartphone or tablet is charged (at least 50%) to avoid power loss mid-session.
- ❑ Insert fresh batteries in your hearing aids. If you have rechargeable hearing aids, make sure they are fully charged.
- ❑ Make sure you choose a location with a data/Wi-Fi connection of 1/1 Mbit/sec or higher. Wi-Fi is recommended where available.
- ❑ Make sure that the Sonic SoundLink Connect is the only running app on your smartphone or tablet. Terminate any other running apps to ensure a smooth connection.
- ❑ Open the Sonic SoundLink Connect app and enter your login credentials. Follow the prompts from the app until you reach the virtual waiting room. Once you are in the virtual waiting room, wait for the app to notify you that your hearing care professional has started the session.

Note: If the connection is unsuccessful, the Sonic SoundLink Connect app will make troubleshooting suggestions. If the connection is still unsuccessful, please contact your hearing care professional.

During your Remote Fitting appointment:

- Sit in a quiet, comfortable place.
- Make sure that your smartphone or tablet is in the proximity of your hearing aids. The smartphone or tablet should not be further away than 16 feet (5 meters) from your hearing aids to keep the connection.
- Make sure that your hearing aids are always on during the Remote Fitting session. Do not turn your hearing aids off.
- Make sure that Bluetooth on your smartphone or tablet is always activated. Do not turn off the Bluetooth.
- Make sure to stay in the Sonic SoundLink Connect app. Do not switch to another app, as this will immediately terminate the appointment.

